interco®

Aktivline!

AKTIVLINE® ECO

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Dynamic seating system

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Standardized dynamic seat and back including wheelchair

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Untergestell
Undercarriage

The AKTIVLINE® ECO model is a standardized care system with all proven dynamic components. It is available in 4 sizes and can be ideally adapted to the needs of the patient through various adjustment options and growth options. The width and depth of the seat and the height of the back are all designed to have 5 cm adjustment intervals, so that they can be simply and rapidly adjusted to the patient's measurements. An anatomical adaption of the seat and back upholstery as well as the adduction guides and an optionally integrated or removable abduction wedge enable perfect positioning of the patient. You can choose between a continuous and a shared footrest system, including various attachments. The new AKTIVLINE® ECO undercarriage is particularly robust and stable and at the same time saves enormous weight while maintaining the same familiar dynamic functions.

- ▶ The entire care unit can easily be removed from the undercarriage and its weight is optimized
- ▶ Dynamic seat system in 4 sizes
- ▶ A variety of additional standardized options
- ▶ New, weight-reduced and robust undercarriage
- ▶ Adjustable seat depth and seat width
- ▶ Continuous or shared footrest system

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- ▶ Infantile cerebral paresis: spastic tetraplegia, dyskinetic tetraplegia, tetraplegia with dystonic spasms, athetosis and chorea-athetosis
- ▶ Cerebral movement impediments.

With this group of patients, sitting as well as the ability to make targeted movements of the head and the upper extremities is very severely restricted as a result of the following patterns:

- ▶ Extension reactions
- ▶ Spasms during extension
- ▶ Pathological movement patterns in global extension
- ▶ Episthotonus
- ▶ Startle reaction
- ▶ Fright reaction
- ▶ Synkinetic extension movements of the trunk, of the upper and lower extremities
- ▶ Dystonic spasms
- ▶ Asymmetric and symmetric tonic reflex of the back of the neck
- ▶ Tonic labyrinthine reflex

Care through the provision of a conventional, rigid seat shell is frequently not possible or very difficult to achieve, due to a lack of acceptance. In particular, the high pressure forces, resulting from conventional sitting care, are impossible for the patient to bear.

In addition, conventional sitting care increases the risk of the occurrence of: hip luxation, painful hips, shortening of the muscles, secondary damage as a result of immobility, muscle contractions or reduced mobility of the joints.

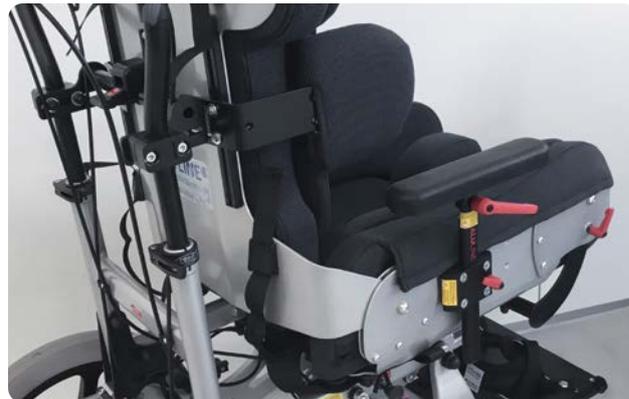
The dynamic seat shell of the AKTIVLINE® seating system enables automatic changing between various body postures and thus encourages breathing, the cardiovascular system, digestion, mobility of the body joints and helps prevent the development of pressure sores.

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AKTIVLINE® dynamic seat-backrest-system made of aluminium with integrated physiological pivots in hip and knee area including:

- ▶ Pelvic retraction unit and system fixation at 90° hip angle
- ▶ Anatomical molded seat with abductionommel with lateral hip support, as well as Velcro side pads (2-layer in 1 cm + 2 cm) for seat width adjustment
- ▶ Anatomical molded backrest with (height and width) adjustable side supports for upper body, with lumbar pad (under cover), with extra backpolder for seat depth adjustment (2 cm)
- ▶ Fabric cover: DOTS Black;
frame color: PEARLDARKGREY RAL 9023
- ▶ Including 5 cm seat depth regulation with continuously adjustable seat plate (3 cm) and option of adjusting the backrest unit (2 cm) and adjustment of the pivot points
- ▶ Ready for fitting armrests
- ▶ Ready for fitting head support and fixation aids
- ▶ Incl. anti-tip and support roller
- ▶ Seating system mount on slide rail with retraction unit and slide rail in the back

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What does “dynamic seat shell” mean?

If we observe a group of children sitting during a school class or in a cinema, we realise that the term “sitting quietly” is more of a wish, which in no way corresponds to reality. Everybody needs some movement even when sitting, in order to improve their own posture, to relieve parts of the body subjected to an excessive period of strain and to keep body joints mobile. AKTIVLINE® is the intelligent solution for all children and adults with a cerebral movement impediment, who cannot control important movement mechanisms independently. Through the dynamic linking of the various parts of the seat unit, it offers corresponding movement of the seat shell. This mobility is adapted to the individual requirements of the user by means of pull and push suspension, with the result that a good sitting position in the interests of therapeutic care in no way interferes with pleasant sitting.

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What advantages does AKTIVLINE® offer the user?

Many people with problems in terms of targeted movement or whose posture is influenced through mainly uncontrollable extension movements, can only benefit from care with the AKTIVLINE® seat shell. In the past, attempts were made to control the involuntary movements, which for various reasons had a negative effect on sitting, by means of firm fixing. However, this type of solution only made the problem worse. The muscle tension, in particular in the hips and trunk, even increased. The consequences are common knowledge: the seat shell often broke or became deformed, the child slid out of the seat shell, the use of wedges or seat pants between the thighs caused increased pressure on the muscles, pressure which is responsible for the luxation of the hips. AKTIVLINE® aims to remove these undesired consequences, and even to offer the possibility of, in part, controlling the movements described, or of using them for a sensible purpose. Thanks to the pull and push suspension, the involuntary movements are permitted to a limited extent. Once the muscles have returned to a relaxed condition, the seat shell turns back to the initial position by itself. Simultaneous physiological and dynamic sitting is achieved. Additionally, the resistance to extension does not – as is the case with conventional forms of care – take maximum effect immediately, but rather not until the end of the movement.

This means that the muscles do not develop maximum force when in a shortened condition (risk of contraction). The result: the perception of one's own body posture is noticeably improved; the possibility exists of in part internalising and controlling it.

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Which children and adults will benefit most from this type of AKTIVLINE® care?

Movement impediments as a result of spastic paralysis (infantile cerebral paresis) can be observed in a wide variety of forms: on the one hand, various parts of the body can be affected (hemiparesis, diparesis, tetraparesis) and, on the other hand, movements themselves can be deficient (akinetik and hypokinetik) or not targeted (diskinetik). The perception component is always of major importance for people in the organisation of their movements. Children with tetraplegia are frequently incapable of controlling their posture and quickly become “frightened” as a reaction to various forms of irritation. An alternative form of care should be tried if over-extension of head, trunk and hips is observed, if movement of the hips is more or less bearable and if almost exclusive use is made of a wheelchair.

Nevertheless, even children who are unable to sit for more than a short time as a result of hip pain, will also feel benefits. For diskinetik movement impediments, AKTIVLINE® offers the only alternative to conventional care. The main advantage lies in the fact that movement components, which disturb target-oriented actions (e.g. speaking, operating a computer), are absorbed by the movement of the shell.

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I am not happy with my rigid seat shell!

Naturally, one should not change satisfactory care through a rigid seat shell under all circumstances. It is known just how long the route to good care can be in some cases. However, if you are not satisfied despite long years of experimenting, because the seat shell frequently breaks, a good sitting posture is not achieved, because pain shortens the sitting period or because involuntary movements have a negative effect on the operation of various devices, then a trial with AKTIVLINE[®] is a recommendable step. It will very soon become apparent whether the advantages of the “dynamic care” are accepted by the user and his/her environment.

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Status as per March 2021
Subject to technical alterations
without prior notification

Managementsystem zertifiziert nach
Management system certified as per
DIN EN ISO 13485:2016
Reg.-Nr. 73 105 6351
TÜV Hessen
DIN EN ISO 9001:2015
Reg.-Nr. 73 100 6351
TÜV Hessen

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© interco Group GmbH, Eitorf, 2021
interco Group GmbH