# MAGICLINE®

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**interco Group GmbH**

Im Auel 50  
53783 Eitorf

Operating instructions  
MAGICLINE®

English

# Contents

---

1.	<b>Introduction</b>	30
1.1.	Product description	30
1.2.	Product overview	30
1.3.	Installation plan	30
1.4.	Environmental conditions	31
2.	<b>Medical requirements</b>	31
2.1.	Field of application/Target group	31
2.2.	Indications	31
2.3.	Contraindications	31
2.4.	Intended use	31
3.	<b>Product labeling</b>	31
4.	<b>Safety instructions</b>	32
5.	<b>Caution symbols</b>	33
6.	<b>Scope of delivery</b>	34
6.1.	Basic configuration	34
6.2.	Options/Accessories	34
6.2.1.	Lower extremities	34
6.2.2.	Pelvis	34
6.2.3.	Upper body	35
6.2.4.	Upper extremities	36
6.2.5.	Head	36
7.	<b>Settings/Operation</b>	37
7.1.	Basic configuration	37
7.1.1.	Attachment and removal of the seat shell via adapter plate	37
7.1.2.	Attachment and removal of the seat shell via locking pin	38
7.1.3.	Adjustment of the back angle	38
7.1.4.	Adaptation of the seat shell to the patient	39
7.2.	Options/Accessories	40
7.2.1.	Pelvis	40
7.2.2.	Upper body	40
7.2.3.	Upper extremities	41
7.2.4.	Head	41
8.	<b>Compatibility/Combination options</b>	42
9.	<b>Technical data</b>	42
10.	<b>Transport/transport lock/climbing stairs</b>	43
10.1.	General information on transport	43
10.2.	Transport with an AMF force application point system	43
10.3.	Transport on aircraft	43
10.4.	Climbing stairs	43
11.	<b>Maintenance plan</b>	43
11.1.	Cleaning	43
11.1.1.	Materials used	43
11.1.2.	Cleaning of materials	44
11.1.3.	Disinfection of the product	44
11.2.	Maintenance	45
11.3.	Service	45
11.3.1.	Seating system	46
12.	<b>Reuse/durability</b>	47
13.	<b>Disposal/environmental compatibility</b>	47
14.	<b>Possible faults</b>	47
15.	<b>EU Declaration of Conformity</b>	48
16.	<b>Warranty policy</b>	49

## Information

Date of last update:  
March 2021

Revision status:  
MAGICLINE\_Gebrauchsanweisung\_interco\_  
de-en\_01

Please read these operating instructions carefully! Observe all safety instructions. If you have any questions, please contact your specialist dealer or interco.

### Note

As a rule, all interco products consist of a seat shell and/or an undercarriage. To simplify matters, only the interco product is mentioned in the following.

In these operating instructions we use the masculine form exclusively for the male/female users of an interco product and its male/female operators (parents, male/female caregivers, male/female therapists).

This way we wish to maintain the readability and comprehensibility of the texts. At the same time, we would like to expressly point out that this naturally includes the female form and other gender identities.

## Foreword

Dear Customer,


Thank you very much for choosing a quality product from interco. This is intended to contribute to a sustainable improvement in the everyday life and quality

of life of the user. To ensure that you can always handle the product safely and use it for a long time, we have described what you should bear in mind during use in these operating instructions.

It is important that you carefully read and understand all sections that apply to your interco product. Keep the operating instructions for future reference. Please follow all the safety instructions and symbols to prevent any type of injury. If you should have any questions about your product and its use, please ask your local qualified medical supplies dealer or contact us directly. We would also be pleased to inform you about further customization and adaptation options for your interco product.

Please observe the following in particular:

- ▶ Parents, accompanying persons, nurses and family members who operate this product should be instructed in the use of the product by authorized specialist personnel using these operating instructions.
- ▶ Your interco product was customized to the needs of the user. Subsequent changes to the product may only be made by trained specialist personnel. We recommend regular checks of the product and the product adaptation to ensure optimal long-term operation.
- ▶ Your interco product is only approved for combination with one of the products listed later on. interco will not accept any liability for combinations with medical devices and/or accessories from other manufacturers.
- ▶ Technical modifications may only be made with the written consent of interco.
- ▶ interco will not accept liability for modifications, changes to parts of the frame of the product and unapproved repairs to this product.
- ▶ interco reserves the right to technical changes to the model described in these operating instructions.

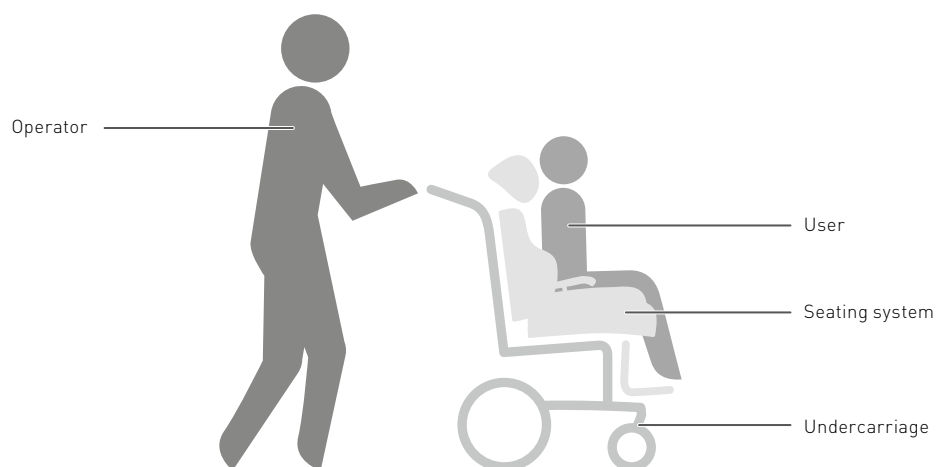# 1. Introduction

## 1.1. Product description

MAGICLINE® is a modular, simultaneously growing seat shell system comprising an aluminium seat and back element; different sizes are possible. That seat system can be adjusted variably within the size specified: The continuous regulation of widths and heights of a seat is effected via longitudinal holes in the aluminium body. The backrest can also be set variably. Comprehensive seat shell accessories like e.g. armrests, headrest, trunk guides, abduction wedge, fixation aids as well as therapeutic desks in different designs can be attached. MAGICLINE® can be mounted for the individual harmonization with the patient's needs with different

underframes for internal and external areas. MAGICLINE® is a seat aid which supports the compensation of a distinct abnormal posture and/or sitting posture instabilities and should mainly be used for children.

An adaptation in growth should additionally be made possible. The design of this active seat system in 3 sizes permits the continuous adjustment of depth, width and height. MAGICLINE® can be mounted on our undercarriages ROOMY®, ROOMY® NEW EDITION, SIMPLY® LIGHT and MINY via trapezoidal adapter. Furthermore MAGICLINE® can be adapted to the most undercarriages of other manufacturers.

## 1.2. Product overview

- 1 Seat unit
- 2 Hip belt
- 3 Lateral side supports
- 4 Chest harness
- 5 Back unit
- 6 Arm rests
- 7 Head rest

## 1.3. Installation plan

### Authorization for delivery

MAGICLINE® may only be delivered to the end customer by authorized specialist dealers, medical device consultants or by the medical device consultants of interco Group.

### Scope of delivery

The scope of delivery includes the basic MAGICLINE® model with the ordered options.

The basic model includes:

- ▶ seat and back unit made of Perldarkgrey
- ▶ physiological pivot points in the pelvic region
- ▶ continuously adjustable seat depths and seat widths using slots
- ▶ variably adjustable back rest
- ▶ continuous back angle adjustment using gas-pressure springs
- ▶ height- and width-adjustable lateral side supports
- ▶ adapter receptacle, trapezoid-shaped, for interco undercarriages
- ▶ cover made of high-quality upholstery fabric

### Setup/assembly

MAGICLINE® is adapted individually to the patient and delivered premounted. After the removal of the transport packing you can use your new MAGICLINE® immediately. If applicable, the accessories or options must be mounted in compliance with instructions in the section 7.

The medical product advisor of the specialist dealer must ensure that the instructions are carried out correctly. (see below) The medical device consultant of interco Group does not have to be present during the installation of the product. However, he can be consulted for support in case of questions.

### Instruction of the customer

All functions of the MAGICLINE® which are explained in the operating instructions must be demonstrated to the customer. The operating instructions remain with the customer. All adjustments to the product that directly affect the user may only be carried out by medical device consultants or in consultation with these.

### 1.4. Environmental conditions

The MAGICLINE® medical device must always be stored in a dry place to prevent possible corrosion. Please

keep to ambient temperatures of -10 °C to +40 °C. Sand or other dirt particles should not be allowed to penetrate the product. This could damage the mechanics. Use in salty water is not permissible.

## 2. Medical requirements

### 2.1. Field of application/Target group

#### Excerpt from the GKV Association

These seating systems can be configured from different units. The basic configuration always consists of a seat and a back in the form of a seat unit. Such modular systems are individually adaptable to the disability due to the large number of different sizes of the respective components, without the need for custom-made solutions.

Seating systems for children are designed for use with a chassis for indoor and outdoor use. Some of these products are already fitted with a chassis as standard.

In many cases, accessories such as neck/head rests, systems for securing and supporting positioning, pads, arm rests and therapy tray tables are required.

Supplying modular child seating systems provides for a favorable seating position and thus provides support and positioning effects for the insured person. These products are intended to straighten the pelvis, stretch the spine as well as to provide lateral support for the pelvis and the thorax.

Due to the many different configuration options, modular seating systems for children are easier to adapt to the size development of children than seating shells, without the need for replacing the care product after only a short time.

### 2.2. Indications

#### Excerpt from the GKV Association

Impaired sitting in the case of functional and/or structural damage to the trunk or trunk muscles and, if applicable, the cervical muscles (e.g. due to neurological/neuromuscular diseases, deformities of the spinal column) with incorrect posture.

To maintain a sitting position which is as physiological as possible and to ensure stable sitting requiring more frequent adjustments as a result of rapid growth during the corresponding developmental phases (particularly in infancy).

*Care area according to the recommendations of § 126 SGB V: 26 B*


### 2.3. Contraindications

There are no known contraindications.

### 2.4. Intended use

MAGICLINE® is a seating aid for children and toddlers that prevents sitting in the event of functional and/or structural damage to the trunk or the trunk and, if applicable, neck muscles. Due to poor posture, users cannot use conventional seating furniture. It is used for therapeutic purposes in order to preserve or expand the motor skills of the user through given freedom of movement and dynamics, as well as to enable active participation in life. The modular seating system can be adapted to the individual needs and disabilities of the user due to the different sizes with the respective add-on parts, without the need for a custom-made product.

## 3. Product labeling

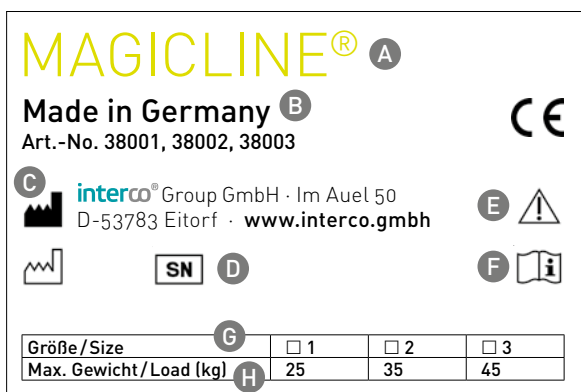- A** Type designation
- B** Country of manufacture
- C** Manufacturer's data
- D** Serial number
- E** Observe warnings/safety instructions
- F** Read operating instructions prior to use
- G** Size
- H** Maximum weight

Please always state the serial number and size of the MAGICLINE® when ordering spare parts or accessories!

## 4. Safety instructions

Please note that you are responsible for the safety of the user. This could be at risk if you do not follow the safety guidelines. Nevertheless, it is not possible to cover all potential circumstances and unforeseeable situations in these guidelines. Therefore common sense, caution and prudence are required when handling the product. Every person operating the product should know and understand all the instructions. Furthermore, you should explain the operation of the product and its accessories to every person using the product.

Please contact your local specialist dealer or interco directly if the instructions are unclear and a more detailed explanation is required or if you have any other questions.

### Meaning of the safety instructions

#### Warning!

Warning against possible severe damage due to accidents or injuries.

#### Caution!

Warning against possible damage due to accidents or injuries.

#### Note!

Warning for possible technical damage.

### Product-specific safety instructions

#### Warning!

*Risk of injury from unattended parking*

If the product is left unattended with user, the user or other persons may be injured.

- ▶ Please make absolutely sure that the person sitting in the product is properly secured at all times by a suitable restraint system (harness).
- ▶ Never leave the user sitting in the product unattended, not even when the user has been securely strapped in and the brakes have been set.
- ▶ Never leave the product alone with the user without expert supervision to exclude the risk of injury to the user or other persons.

#### Warning!

*Risk of suffocation due to the packaging material*

Incorrect handling of packaging material may cause a risk of suffocation.

- ▶ Keep the packaging materials out of reach of children.

#### Warning!

*Risks when climbing stairs*

When using stairs, there may be a risk of injury to the user of the product or the product may be damaged. Therefore only lift the product without the user sitting in it.

- ▶ Stairs are only permitted to be negotiated with the assistance of accompanying persons.
- ▶ If facilities such as access ramps or lifts are available for this purpose, these must be used. In the absence of such facilities, two accompanying persons can carry the product over obstacles without the user.
- ▶ Techniques for climbing stairs are explained in detail in Section 10; it is imperative that you comply with these.
- ▶ Make sure that the accompanying persons only grip the product on firmly mounted or bonded components.
- ▶ Do not lift the product by the foot rest, head rest, arm rests or similar parts.

#### Warning!

*Risks of self-initiated modification of the settings*

If settings are changed autonomously, the function of the product and the safety of the user may be affected.

- ▶ Adhere to the settings made by the specialist dealer. You may only adjust settings by yourself as described in Section 7.
- ▶ If you experience problems with settings, please contact your specialist consultant or interco directly.
- ▶ Discuss all changes to the settings properly with the specialist personnel or therapist.
- ▶ It is not permissible for any parties other than interco to make additions or modifications to the product and any cases that should become known can be subject to legal action. In addition, the warranty obligations of interco Group shall cease to apply in such cases and any liability claims directed at the manufacturer shall no longer be valid.

#### Caution!

*Risk of injury due to operating errors or lacking knowledge of the operating instructions*

If the instructions in the operating instructions are disregarded, application errors may occur and the safety of the user may be impaired. Please read these operating instructions carefully first.

- ▶ Familiarize yourself with the functions of the product and its handling prior to initial use. Practice handling the product before use.
- ▶ Observe the maximum load capacity of the product (see Section 9: Technical Data).
- ▶ Do not use the foot rest when getting into or out of the product.

**Caution!**

*Risk of tipping due to incorrect center of gravity / shift in center of gravity*

If the product's center of gravity is shifted significantly, for example on slopes and inclines or if the user leans out of the product too far, the product may tip or roll over.

- ▶ Test the effects of shifts in the center of gravity on the behavior of the product, for example, on declines, inclines, and lateral slopes or while negotiating obstacles only with the reliable support of an assistant.
- ▶ Always stand the product on horizontal, flat surfaces only. If it is unavoidable to park it on an incline, put the seat in an upright position. There is a risk of the product tipping backwards on inclines in the lying position.
- ▶ When grasping objects (lying in front of, to the side or behind the device) make sure that the user does not lean out of the seat too far.
- ▶ Do not hang heavy shopping bags or similar on the push bar or the levers.

risks of accidents and injuries for the user and the operators.

- ▶ Check all screws and nuts regularly and retighten them.
- ▶ Alternatively, you can also instruct your specialist dealer or interco to do this.

**Caution!**

*Dangers when making adjustments to the product and connections*

If you do not carry out adjustments to the product properly, the stability and safety of the product may be affected.

- ▶ Ensure that the screws are tightened again after all adjustment and assembly work.
- ▶ Always make sure that the seating system is securely connected to the undercarriage.
- ▶ If you have a question regarding the connection of the seating system and the undercarriage, always contact your specialist dealer or interco directly.
- ▶ Carry out a regular visual inspection of all connections of the product. Consult your local specialist dealer in case of doubt.

**Caution!**

*Risks in public road traffic*

Dangerous situations can occur when driving on public roads.

- ▶ Please note that you must observe traffic regulations when traveling on public streets.
- ▶ In dark conditions wear bright clothing wherever possible in order to be more visible. We also recommend installing an active light source.

**Caution!**

*General risks due to careless handling of the product*

The product is only approved for its intended use. Improper or careless use can result in safety and injury risks for the user and other persons.

- ▶ Avoid actions, such as driving against an obstacle (stair, curb) without using the brake or down from elevated surfaces.
- ▶ Use caution when handling fire (e.g. a burning cigarette)! Upholstery and coverings could catch fire.
- ▶ Make sure to keep children from playing with the therapy chair or any parts thereof. There is a risk of injury due to trapping and crushing hands and fingers.
- ▶ Children must not play next to, underneath or in front of the seating system when the seating system is in motion and/or the seat or back unit is in use.

**Caution!**

*Risk of injury to hands and fingers*

Your hands or fingers can get caught during adjustment and assembly work.

- ▶ Always be aware of your fingers during operation and all adjustment and assembly work.

**Caution!**

*Dangers due to loosened components during use*


Screws and nuts on the product may become loose when the product is used. This can result in

## 5. Caution symbols

**Note:** all screw connections that were loosened during adjustment must be firmly retightened.

**Place of mounting:** Back shell edging left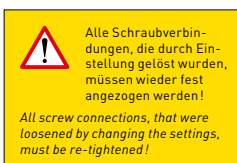

## 6. Scope of delivery

### 6.1. Basic configuration

- MAGICLINE® is a modular, adjustable seating system with
- ▶ seat and back unit made of aluminum in Perldarkgrey
  - ▶ physiological pivot points in the pelvic region
  - ▶ continuously adjustable seat depths and seat widths using slots
  - ▶ variably adjustable back rest
  - ▶ continuous back angle adjustment using gas-pressure springs
  - ▶ height- and width-adjustable lateral side supports
  - ▶ adapter receptacle, trapezoid-shaped, for interco undercarriages
  - ▶ cover made of high-quality upholstery fabric

### 6.2. Options/Accessories

#### 6.2.1. Lower extremities

Artikel	size	Art. no.
ALULINE leg rest system with incrementally adjustable holder for leg rest system Foot rest, complete version, foot rest powder coated in black, includes corner guards and foot rest overlay for all interco undercarriages	size 1-3	46013
ALULINE leg rest system with incrementally adjustable holder for leg rest system Foot rest with individual foot supports, foot rest powder coated in black, includes corner guards and foot rest overlay for all interco undercarriages	size 1-3	46016
Leg rest, size 1, assembly kit with 90° fixed foot bracket with holder for leg rest system with continuous angle adjustment; with foot rest overlay for MAGICLINE®, for minimal lower leg length	size 1-2	38221-NL 38222-NL
Leg rest assembly kit with fixed foot bracket and holder for leg rest system fixed at 90°	size 1-2	38211 38212
Leg rest assembly kit with fixed foot bracket with continuous angle adjustment	size 1-2	38221 38222
Foot shells made of aluminum	size 1-3	46210
Leg rest cushion, including black synthetic leather cover		38050
Foot rest riser made of black plastic		46530
Heel stop for leg rest system, complete version		46551
Heel stop for leg rest with individual foot supports		46552
Heel stop cushion, including black synthetic leather cover		38040
Calf support, complete version, includes black synthetic leather cover		38030
Calf support, single version, includes black synthetic leather cover		38020
Foot straps with hook and loop fastener with padded instep		46410
Foot straps with deflection roller and padded instep with hook and loop fastener		46412
Foot straps with incremental adjustment and padded instep		46419
Foot straps with clip buckle and padded instep		46420
Foot gaiters without forefoot restraint made of neoprene	size 1-3	46414
Foot gaiters with forefoot restraint made of neoprene	size 1-3	46416

#### 6.2.2. Pelvis

Artikel	size	Art. no.
Abduction pommel, size 1 for MAGICLINE® with holder, adjustable and attachable and pommel upholstered in fabric; MEASUREMENTS: D = 140 mm; H = 80 mm; W = 80 mm complete also as upgrade kit including mounting materials		38111

Artikel	size	Art. no.
Abduction pommel, size 2 for MAGICLINE® with holder, adjustable and attachable and pommel upholstered in fabric; MEASUREMENTS: D = 160 mm; H = 90 mm; W = 90 mm complete also as upgrade kit including mounting materials		38112
Abduction pommel, size 3 for MAGICLINE® with holder, adjustable and attachable and pommel upholstered in fabric; MEASUREMENTS: D = 190 mm; H = 110 mm; W = 110 mm complete also as upgrade kit including mounting materials		38113
Abduction pommel, size 1 for MAGICLINE® with holder, swing-away, and pommel upholstered in fabric; MEASUREMENTS: D = 100 mm; H = 80 mm; W = 80 mm complete also as upgrade kit including mounting materials		38114
Abduction pommel, size 2 for MAGICLINE® with holder, swing-away, and pommel upholstered in fabric; MEASUREMENTS: D = 160 mm; H = 90 mm; W = 90 mm complete also as upgrade kit including mounting materials		38115
Abduction pommel, size 3 for MAGICLINE® with holder, swing-away, and pommel upholstered in fabric; MEASUREMENTS: D = 190 mm; H = 110 mm; W = 110 mm complete also as upgrade kit including mounting materials		38116
2-point hip belt, size 1, with neoprene cushion and small plastic press buckle with one pull		44175-1
2-point hip belt with neoprene cushion and large plastic press buckle with one pull	size 2-3	44175-2 44175-3
4-point hip belt, size 1, with neoprene cushion and small plastic press buckle with one pull		44176-1
4-point hip belt with neoprene cushion and large plastic press buckle with one pull	size 2-3	44176-2 44176-3
2-point hip belt with neoprene cushion with two pulls	size 1-3	44177
4-point hip belt with neoprene cushion with two pulls	size 1-3	44178
Pelvic harness standard, leg-spreading or T-shaped in spacer fabrics, synthetic leather or neoprene	size 1-3	44190 44192 44198

### 6.2.3. Upper body

Artikel	size	Art. no.
Lumbar spine support with Velcro attachment under the cover	size 1-3	38121 38122 38123
Lateral side supports, size 1, for MAGICLINE® with ALULINE bracket, includes swing-away unit upholstered in fabric; MEASUREMENTS: D = 28 mm; H = 100 mm; W = 100 mm (also as upgrade kit including mounting materials)		38131
Lateral side supports, size 2, for MAGICLINE® with ALULINE bracket, includes swing-away unit upholstered in fabric; MEASUREMENTS: D = 28 mm; H = 120 mm; W = 120 mm (also as upgrade kit including mounting materials)		38132
Lateral side supports, size 3, for MAGICLINE® with ALULINE bracket, includes swing-away unit upholstered in fabric; MEASUREMENTS: D = 28 mm; H = 140 mm; W = 140 mm (also as upgrade kit including mounting materials)		38133
Holder for OFF-SET version of the lateral side supports	size 1-3	38160
Holder for OFF-SET version of the lateral side supports including swing-away unit	size 1-3	38161
Chest belt with deflection roller in spacer fabrics, synthetic leather or neoprene		44110u
Chest belt, basic version (with Velcro attachment) made of synthetic leather or neoprene	size 1-3	44114
Chest belt with flap made of synthetic leather or neoprene	size 1-3	44115
Standard RV chest harness with push-in buckle, 25 mm, lock-stitched straps on the shoulders and zipper in the middle (includes clip buckle with fastening tab for mounting on seat shells)	for boys or girls in spacer fabrics, synthetic leather or neoprene size 1-4	47130

Artikel		size	Art. no.
VARIO chest harness with twist-lock at the pelvis and push-in buckles on the upper shoulder straps (includes clip buckle with fastening tab for mounting on seat shells)	for boys or girls in spacer fabrics, synthetic leather or neoprene	size 1-4	47131
REMOVABLE chest harness with removable upper straps with push-in clasp and push-in clasps at the pelvis	for boys or girls in spacer fabrics, synthetic leather or neoprene	size 1-4	47132
Standard chest harness with push-in buckle, 25 mm, lock-stitched straps on the shoulders (includes clip buckle with fastening tab for mounting on seat shells)	for boys or girls in spacer fabrics, synthetic leather or neoprene	size 1-4	47133
Strap guide plates (includes clip buckle compatible with straps) for mounting on the edges of the seat shell, used to guide upper shoulder strap and enable continuous height adjustment			47200

#### 6.2.4. Upper extremities

Artikel		size	Art. no.
ALULINE arm rests, incrementally height- and angle-adjustable			38140
Height-adjustable arm rests (15x15 mm square) including PUR arm cushions			42010
Height- and angle-adjustable arm rests (15x15 mm square) including PUR arm cushions			42020
Therapy tray table made of acrylic, square-shaped, with upturned front and side bars for MAGICLINE®		size 1-3	38191 38192 38193
Therapy tray table made of beech wood, square-shaped, with upturned front and side bars for MAGICLINE®		size 1-3	38194 38195 38196
Attachment mechanism for therapy tray table for attachment with two brackets			38197
Swing-away unit for therapy tray table			38198

#### 6.2.5. Head

Artikel		size	Art. no.
ALULINE head rest bracket, fixed model, with support block, includes 1 x ALULINE universal bracket with one or two fastening elements			38101 38102
ALULINE head rest bracket, laterally adjustable model, with support block, includes 1 x ALULINE universal bracket with one or two fastening elements			38103 38104
Standard head rest cushion; color: black synthetic leather for MAGICLINE® sizes 1-2 including fabric cover		size 1-2	38171 38172
Head rest cushion, shell-shaped; color: black synthetic leather for MAGICLINE® sizes 1-2 including fabric cover		size 1-2	38173 38174

## 7. Settings/Operation

Every interco product is customized to the individual needs of the user and configured accordingly. To do justice to all possible product designs, the list of options and accessories as well as their descriptions is therefore somewhat more extensive.

Therefore please bear in mind that this section may also describe components that possibly do not exist on your product.

### 7.1. Basic configuration

#### *Tool*

As standard, the MAGICLINE® is adjusted with an Allen wrench. This tool can be used to perform all adjustments to the MAGICLINE®.

#### **Important!**

Retighten all screws again when adjustment has been completed!  
Make sure there is enough space (approx. 2 cm) in the back of the knees!

#### 7.1.1. Attachment and removal of the seat shell via adapter plate

##### *Attaching the seat shell*

The seat shell is fixed to the adapter plate of the underframe or wheel chair by means of the trapezoidal adapter (Fig. 1). For that purpose perform the subsequent steps and check the firm fit on the adapter plate after the seat shell has been placed.

- ▶ First unlock the seat shell fixation (Fig. 2).
- ▶ Set the rear end of the seat shell adapter to the front of the adapter plate such that the locking is pressed downward (Fig. 3).
- ▶ Then push the seat shell backwards until it engages audibly.
- ▶ Lock the seat shell fixation again (Fig. 4).
- ▶ Finally, check the seat shell and the seat shell catch for firm fit (Fig. 6).

#### **Important!**

The example shows the trapezoidal adapter of the underframes of the ROOMY® NEW EDITION. Differences between this adapter and your item may occur.

#### **Important!**

The placing of the seat shell on the wheel chair may only be performed without patient.

### Removing of the seat shell

First loosen the protective device (Fig. 5). Open the seat shell catch by pressing the lever upwards (Fig. 6). Then you can pull out the seat shell to the front.

#### Important!

The removal of the seat shell from the wheel chair may only be performed without patient.

#### Important!

The example shows the trapezoidal adapter of the underframes of the ROOMY® NEW EDITION. Differences between this adapter and your item may occur.

### 7.1.2. Attachment and removal of the seat shell via locking pin

In order to fix the seat shell please follow these steps:

1. Check whether the release button is locked in. If yes, the locking pin should reach out on top of the adaptor plate. If not, turn the locking pin until it locks.
2. Position the seat unit with the rear part of the trapezoidal adaptor on the trapezoidal plate. Push the seat unit (and thereby the trapezoidal adaptor) backwards into the trapezoid plate until the locking pin engages audibly

#### Important!

Check that the seat shell and the lock are firmly fixed by pulling the seat while pushing the undercarriage. The seat shell should not slide out.

To remove the seat shell proceed as follows:

- ▶ Pull the locking pin downward and turn by 90°. This way the locking pin will remain in its unlocked position.
- ▶ Open the seat shell fixation by pushing down the lever. Now you can pull the seat shell out towards the front.
- ▶ Remove the seat shell from the undercarriage.
- ▶ Turn the locking pin until it locks audibly.

### 7.1.3. Adjustment of the back angle

The rear angle can be adjusted comfortably by means of the gas pressure spring on the rear of the seat shell (Fig. 7).

Press the lever at the lower end of the gas pressure spring and manually set the rear angle continuously (Fig. 8). When the lever is released, the setting angle is fixed.

Fig. 5

Fig. 6

Fig. 7

Fig. 8

#### 7.1.4. Adaptation of the seat shell to the patient

##### Important!

Alterations as to fit, clinical picture or the like may only be performed by the competent specialized company!

##### Adjustment of width of seat

Loosen the four setscrews for the adjustment of the seat width on the lower side of the seat shell (Fig. 9) by means of a hexagon socket screw key such that the connection can be moved. Also loosen the setscrews on the back of the seat shell, two pieces each at the bottom on each side (Fig. 10). Then you can set the width of seat to the dimension desired. Following that, tighten the screws again.

##### Important!

Upon completion of the setting works check all connections for firm fit again.

##### Adjustment of depth of seat

Loosen the setscrews for the adjustment of depth of seat at the bottom of the seat shell (Fig. 11) by means of a hexagon socket screw key just so far that the connection can be moved. Then loosen the bur-type connection of the seat upholstery, and then you can set the depth of seat to the dimension desired.

Tighten the screws again and fasten the seat upholstery by means of the bur-type tape. Upon completion of all setting works check all connections for firm fit.

##### Important!

Alterations as to fit, clinical picture or the like may only be performed by the competent specialized company!



##### Adjustment of backrest height

Loosen the four setscrews for the adjustment of the height of seat on the rear of the seat shell (Fig. 12) by means of a hexagon socket screw key just so far that the connection can be moved. Then loosen the bur-type connection of the back upholstery.

Following that, you can set the height of seat to the dimension desired and tighten all screws.

##### Important!

Upon completion of all setting works check all connections for firm fit.

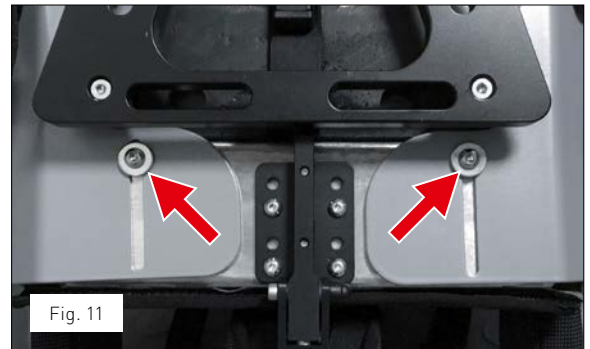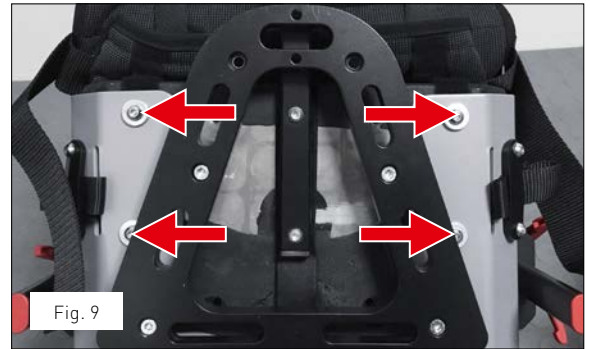### Adjustment of the lateral side support

The lateral side support can be adjusted as to height as well as to their lateral distance to the patient. The setscrews are found on the back of the seat shell, two each at each pressure pad. Loosen the setscrews (Fig. 13) by means of a hexagon socket screw key just so far that the connection can be moved (Fig. 18). Then you can set height and lateral distance desired and tighten the screws again.

#### Important!

Upon completion of all setting works check all connections for firm fit.

## 7.2. Options/Accessories

### 7.2.1. Pelvis

#### Safety belt for the pelvis

MAGICLINE® can be equipped with a belt for the pelvis at a 45°-angle as well as a 70°-angle.

A belt is mounted at a 45°-angle after a belt holding device has been mounted to the rear threaded bore and the belt is inserted between backrest and lateral parts (Fig. 14).

Mounting at the 70°-angle is performed on the lower side of the lateral parts after a belt holding device has been mounted between the two threaded bores and the belt is inserted through the guiding slots provided on the lower side of the lateral parts and moved upwards between lateral parts and seat area (Fig. 15).

#### Abduction wedge

Abduction wedges can be obtained in two designs for MAGICLINE®, rigid and swivelling. Both are plugged onto the holding device (square pipe) to the lower side of the seat shell and fixed by means of a fixing lever (Fig. 16).

### 7.2.2. Upper body

#### Belt for the breast

To mount a belt for the breast MAGICLINE® is provided with a mount on the lateral side support. Fasten the belt for the breast to the threaded bores provided.

#### Chest harness

MAGICLINE® can be equipped with different Chest harnesses. Fastening is identical with the different designs.

The lower belts of the pressure pads for breast and shoulders are fastened to the fastening elements of the lateral parts on the rear of the seat shell and moved to the front between backrest and seat area (Fig. 25). At the top, the belts are inserted into the tilting buckle on the rear of the seat shell and fixed by the tilting buckle (Fig. 17).

Fig. 13

Fig. 14

Fig. 15

Fig. 16

For correct positioning of the patient the chest harness is loosened at the connecting elements (Fig. 26) to lay it over the patient. To adapt the chest harness as to length, use the tilting buckle at the backrest (Fig. 27).

### Lumbar spine support

MAGICLINE® can be equipped with a velcro-type lumbar spine support.

Simply fasten it by means of a velcro connection below the cover of the backrest at the therapeutically required height.

### Lateral side supports

For this item two designs can be obtained for assembly to MAGICLINE®, a rigid one and a design with swing-away holder. Both are fastened to the mount rails on the back of the seat shell by means of two hexagon socket screw keys [Fig. 18].

### Swing away the lateral side support

Loosen the catch by pressing the red button on the side of the pressure pads. Then let the lateral side support swing away. When the lateral side support swing back, the catch is engaged audibly and thus the lateral side support is fixed.

## 7.2.3. Upper extremities

### Armrests

MAGICLINE® can be equipped with different armrests. Fasten the armrest by loosening the fixing lever at the armrest mount (Fig. 19/1) on the right and left on the sides of the seat shell, inserting the headrest holding device and tightening the lever again.

The height of the arm rest can be adjusted using the lower clamping lever on the universal holder. (Fig. 19/2) The inclination of the arm rest can be adjusted via the ratchet of the upper clamping lever.

### Important!

Always release only one clamping lever and make the adjustment here. Only continue with the next one after the previous one has been secured.



## 7.2.4. Head

### Headrest systems

MAGICLINE® can be equipped with different headrest pressure pads. The example shows the ALULINE headrest system.

Fasten the headrest by loosening the fixing lever at the headrest mount found at the upper part of the backrest (Fig. 20), inserting the headrest holding device and tightening the lever again.

The same method is used when fixing the head rest with a wing screw. First the screw is loosened, the head rest is adjusted, finally the screw is tightened once again.

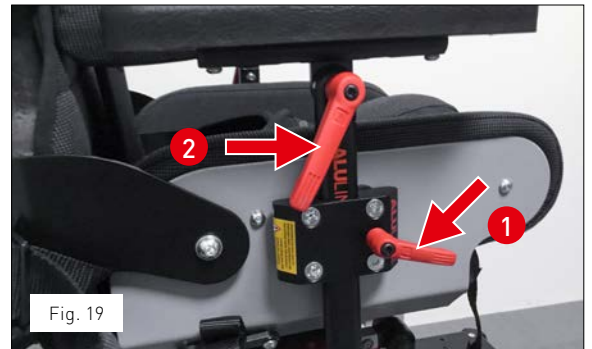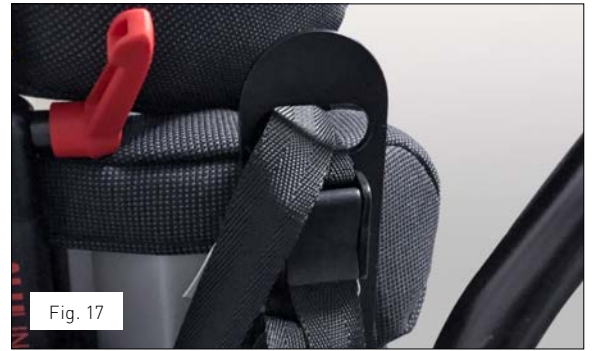## 8. Compatibility/Combination options

Undercarriage	MAGICLINE® size 1	MAGICLINE® size 2	MAGICLINE® size 3
SIMPLY SB 45 · 22"/24" wheels	-	-	✓
SIMPLY SB 45 · 12"/16" wheels	-	-	✓
SIMPLY SB 40 · 22"/24" wheels	-	✓	✓ without wheel cover
SIMPLY SB 40 · 12"/16" wheels	-	✓	✓
SIMPLY SB 35 · 22"/24" wheels	✓	✓ without wheel cover	-
SIMPLY SB 35 · 12"/16" wheels	✓	✓	-
SIMPLY LIGHT SB 45 · 22"/24" wheels	-	-	✓
SIMPLY LIGHT SB 45 · 12"/16" wheels	-	-	✓
SIMPLY LIGHT SB 40 · 22"/24" wheels	-	✓	✓ without wheel cover
SIMPLY LIGHT SB 40 · 12"/16" wheels	-	✓	✓
SIMPLY LIGHT SB 35 · 22"/24" wheels	✓	✓ without wheel cover	-
SIMPLY LIGHT SB 35 · 12"/16" wheels	✓	✓	-
ROOMY NEW EDITION · 12"/16" wheels outdoor	✓	✓	✓
ROOMY NEW EDITION · 100/125 mm wheels indoor	✓	✓	✓
MINY seat unti frame SB40 · 22"/24" wheels	-	✓	✓
MINY seat unti frame SB40 · 12"/16" wheels	-	✓	✓
MINY seat unti frame SB35 · 22"/24" wheels	✓	✓	✓ without wheel cover
MINY seat unti frame SB35 · 12"/16" wheels	✓	✓	✓
MINY seat unti frame SB30 · 22"/24" wheels	✓	-	-
MINY seat unti frame SB30 · 12"/16" wheels	✓	✓	-

## 9. Technical data

### Caution!

The maximum load (workload) for MAGICLINE® is 25 kg for size 1, 35 kg for size 2 and 45 kg for size 3.

MAGICLINE®	Size 1	Size 2	Size 3
Weight without patient	approx. 8 kg	approx. 9 kg	approx. 10 kg
Max. load	25 kg	35 kg	45 kg
Max. patient height	120 cm	135 cm	150 cm
Overall exterior width	30-35 cm	35-40 cm	40-45 cm
Overall exterior depth	42 cm	47 cm	52 cm
Total exterior height	44-50 cm	48-55 cm	53-60 cm
Back angle adjustment	-5° - 30°	-5° - 30°	-5° - 30°
Seat width	20-25 cm	25-30 cm	30-35 cm
Seat depth	20-25 cm	25-30 cm	30-35 cm
Back height	34-40 cm	39-46 cm	44-51 cm

## 10. Transport/transport lock/climbing stairs

### 10.1. General information on transport

The MAGICLINE® product in combination with a TÜV approved interco undercarriage is permissible as a seat in a motor vehicle.

#### Warning!

*Dangers due to incorrect fastening of the transport safety device*

If you attach the safety device for transport in a vehicle to the wrong places on the product, it cannot be transported safely.

- ▶ Never secure the transport safety device to the frame or other unsuitable basic components!

### 10.2. Transport with an AMF force application point system

This option is only permissible for MAGICLINE® with an interco undercarriage.

### 10.3. Transport on aircraft

When traveling by air, the product may only be transported without the user in the cargo hold of the aircraft.

There are no further instructions or regulations for transport in an aircraft. We therefore strongly recommend that you contact the airline before traveling by air to clarify the transport options in each individual case.

#### Caution!

*Transport on aircraft*

The user must never be in the device when it is transported in the cargo hold of an aircraft!

### 10.4. Climbing stairs

Stairs or high curbs can only be negotiated with the assistance of two accompanying persons. Assistants are only permitted to grip the front frame tube and the back brace of the frame, without lifting or carrying the product when doing so.

When ascending, the assistant behind the device pulls the device up the step. The assistant in the front stabilizes the position by pressing the device equally on both sides into the step.

When descending, the assistant at the front brakes by pressing the device evenly into the steps. The assistant behind holds the back brace, and stabilizes and maintains the device in the correct position. To prevent damage, slowly roll the product down the steps. Then properly readjust the anti-tipping system (if installed).

#### Caution!

*Risks when climbing stairs*

If you hold or lift the product by its attachments when climbing stairs, these may tear off and the user may be injured.

- ▶ Do not hold the product by the push bar when negotiating stairs.
- ▶ Never lift the product by the foot rest, the wheels or the arm rests or other attached parts.
- ▶ Do not lift the product with the user sitting in it.

## 11. Maintenance plan

### 11.1. Cleaning

Your interco product is made of high-quality and durable materials which are easy to clean. Please observe the following instructions for long durability of the product:

- ▶ Always keep the product dry, at temperatures between -10 and +40 °C. This prevents corrosion.
- ▶ Sand or other dirt particles should not be allowed to penetrate the product. This could damage the mechanics or - if applicable - the wheels.
- ▶ Never use the product in salt water.
- ▶ The cushions can be removed for cleaning and care.

#### 11.1.1. Materials used

interco uses the following materials in the manufacturing of medical devices and accessories:

- ▶ Metals (aluminum, steel, stainless steel)
- ▶ Plastics (OPM, PA6, PTEG)
- ▶ Beech wood (for therapy tray tables)
- ▶ Foams (frame foam, polyurethane foam, hard foam, azote foam, neoprene)
- ▶ Upholstery fabrics (mesh material, Trevira CS fabrics, artificial leather, Dartex)
- ▶ Powder coatings
- ▶ DIN and standard parts

All materials used are classified as harmless to health.

#### Powder-coated aluminum

interco undercarriages and some add-on parts are made of aluminum and feature a resistant plastic coating. Powder varnishes are used for coating. Powder varnishes are solvent-free coating materials, composed of different resin systems, pigments as well as special additives.

#### Fabrics

interco uses different fabrics for covers and fixation aids. All materials used are flame-retardant.

#### Synthetic leather

interco uses easy-care designer synthetic leather for Dynaline restraints. This synthetic leather is long-lasting, hard-wearing, very elastic and tear-resistant. It provides a good grip and offers a comfortable seating experience. In addition, it is resistant to urine, blood and disinfectants, free of azo compounds, CFC, formaldehyde, as well as free of PCTs, PCPs and PCBs.

### Spacer fabrics/mesh material

The spacer fabrics used have two outer textile layers, which are connected by a number of spacer threads. This 3D structure creates an insulating layer of air between the textile layers, which balances temperature fluctuations effectively. Furthermore, the spacer fabrics are very breathable, pressure-elastic, light and tear-resistant.

### Supra-tex

The SUPRA-tex upholstery fabric consists of highly tear-resistant polyamide. It is impregnated to be water-repellent, PU-coated on the back and flame-retardant.

### Neoprene

The neoprene used is a foam based on chloroprene rubber (CR). It insulates very well, is waterproof and very well tolerated by the skin. In addition, the neoprene is tear and abrasion resistant, very elastic, resistant to fungi and bacteria and eco-friendly.

### Trevira CS (fabrics)

Among others, interco uses Trevira CS fabrics for seat covers. These are easy to clean, dry quickly and keep their shape. In addition, Trevira CS can be disinfected under hospital conditions.

### Dartex incontinence cover fabric

interco uses the incontinence-resistant upholstery fabric Dartex for incontinence-resistant seat shells. The high-quality fabric meets all quality and safety requirements and is easy to wash off, sufficiently lightfast and flame-retardant. It does not cause allergies, is free of formaldehyde and azo dyes.

## 11.1.2. Cleaning of materials

### Powder-coated aluminum

Clean undercarriages and attachments made of powder-coated aluminum with mild soapy water and a soft sponge. Do not use aggressive or abrasive agents. These can damage the surface of the coating.

### Fabrics

Clean the fabric covers regularly before stains set in. You can vacuum or carefully brush the textiles. Spilled liquids should be wiped up immediately with an absorbent cloth. You can also put a mild detergent solution on a cloth, dab the stain with it and then rub off gently with clear water.

#### Note!

Bleach or whiteners should never be applied to the fabrics / covers. Never dry the covers in the dryer! Only iron at a low temperature.

If you wish to wash the cover of the seating system more often, a second cover for changing is recommended. Ask your local specialized dealers on this topic.

### Synthetic leather

Clean synthetic leather covers with a cloth and commercially available mild soap solution.

### Spacer fabrics/mesh material

Wash removable covers made of spacer fabrics/mesh material at a maximum of 30 °C in the washing machine with the delicates program.

### Supra-tex

interco uses Supra-tex for the arm rests. Clean these regularly with a cloth and commercially available mild soapy water.

### Neoprene

Wash removable fixation aids made of neoprene at a maximum of 30 °C in the washing machine. Use a mild liquid detergent or a commercially available neoprene detergent. Use a gentle wash cycle and do not spin-dry the neoprene covers. Then hang up the covers wet to dry.

### Trevira CS (fabrics)

Removable covers made of Trevira CS can be dry cleaned. Washing under easy-care conditions at 60°C is possible. Do not dry and do not iron!

### Dartex

Test washed by the manufacturer at temperatures between 40 and 95 °C.

Accessory components made of metal, plastic, wood  
Please clean accessory components with a conventional mild household soap solution.

## 11.1.3. Disinfection of the product

Please use a water-based agent for disinfection of the product, e.g. Terralin, Quartamon Med or Sagrotan Original concentrate. Please observe the manufacturer's instructions for use when doing so. When disinfecting the fabrics, also follow the instructions of the fabric manufacturer regarding the use of brand name disinfectants.

Rinse off all detergents and disinfectants thoroughly and leave the cleaned object to dry completely before use. If this is neglected, this can lead to chemical reactions making the material brittle or lead to skin irritations of the patient.

#### Important!

*No disinfectants for synthetic leather/skai*  
Synthetic leather covers/skai covers may not be cleaned with disinfecting agents, as the material gets porous over time and cracks.

#### Note!

*Wipe disinfection*  
If wipe disinfection is necessary, use a disinfectant that is suitable for surface disinfection according to the DGH disinfectant list.

## 11.2. Maintenance

Your interco product is checked by trained and qualified employees for proper functioning and freedom from defects before being handed over to the user. To ensure safe use of the product over its entire service life (according to the manufacturer from at least 2 years to 6 years maximum) and to ensure a perfect technical condition, it must be maintained and inspected regularly according to the following maintenance (every 6 months) and service (every 12 months) plans.

### Note!

#### Changing settings

The changing of settings (other than those described in Section 7) is only permitted for qualified and trained personnel!

Incorrect settings by the user, operator or laypersons bear the risk of injuries or crushing, which in a worst case scenario could lead to the patient's death!

During normal use, you should regularly check the interco product for damage and the perfect working condition of all components.

### Important!

#### Original spare parts

Only original parts may be used for service and maintenance work.

Checking / components	Daily	Weekly
Visual inspection of the seating system and the attached components and accessories for visible damage	x	
Check of connection between undercarriage and seating system	x	
Visual inspection of all fastening elements	x	
Visual inspection of all accessories / options	x	
Retightening of all fastening elements		x
Retightening of all screw connections of the accessories / options		x

## 11.3. Service

Test / Execution	OK	Adjustments necessary
Cleanliness	<input type="checkbox"/>	<input type="checkbox"/>
Check of all settings of the MAGICLINE® seating system	<input type="checkbox"/>	<input type="checkbox"/>
Check of setting of the head rest:	<input type="checkbox"/>	<input type="checkbox"/>
▶ Does the head rest lie against the patient's occiput (standard version)?		
▶ Does the bottom edge of the head rest lie against the base of the head (shell shape)?		
▶ Do the side elements exert pressure on the patient's head?		
▶ Does the patient have sufficient freedom of movement?		
Check of setting of the foot rest:	<input type="checkbox"/>	<input type="checkbox"/>
▶ Is there enough free space at the back of the knee?		
Check of setting of the arm rests:	<input type="checkbox"/>	<input type="checkbox"/>
▶ Are the shoulders in a neutral / ergonomic position when the elbows are in the resting position?		
Check of setting of the lateral side supports:	<input type="checkbox"/>	<input type="checkbox"/>
▶ Is there enough free space under the armpits?		
▶ Is the upper body guided sufficiently?		
Check the safety distances according to DIN EN 12182:2012 using the following table	<input type="checkbox"/>	<input type="checkbox"/>

### Safety distances according to DIN EN 12182:2012 for children

To avoid	Between moving parts	Between fixed parts
Trapping fingers	<4 mm or >25 mm	<5 mm or >12 mm
Trapping feet	<25 mm or >120 mm	<25 mm or >45 mm
Trapping the head	<60 mm or >300 mm	<60 mm or >250 mm
Trapping the genitals	<8 mm or >75 mm	<8 mm or >75 mm

### Safety distances according to DIN EN 12182:2012 for adults

To avoid	Between moving parts	Between fixed parts
Trapping fingers	<8 mm or >25 mm	<8 mm or >25 mm
Trapping feet	<35 mm or >120 mm	<35 mm or >100 mm
Trapping the head	<120 mm or >300 mm	<120 mm or >250 mm
Trapping the genitals	<8 mm or >75 mm	<8 mm or >75 mm

**Note!**

*Expiry of the manufacturer's warranty if service is not conducted*

The product must be checked regularly – at least once a year – as part of a (payable) service.

Otherwise interco Group cannot provide any warranty.

### 11.3.1. Seating system

Check of setting / function	OK	Damage/ distortion	Exchange/ replacement
<b>Function check</b>			
Back adjustment	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Fixing unit	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Arm rest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Head support	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Abduction wedge	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Therapy table	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>Screw connections</b>			
Screws tightened	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Rust spots	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>Appearance (visual inspection)</b>			
Paintwork	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Cleanliness	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Fabric seams (position / appearance)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Type plate	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

## 12. Reuse/durability

---

Your product can be reused under certain conditions. It is CE marked and must be reconditioned for reuse by the manufacturer to meet all essential safety and performance requirements. This includes that it is properly adapted to the needs of the new user. In particular, individual fittings can only be reinstated after professional adaptation to the new patient. Accompanying documents, as well as these operating instructions, are part of the product and must be handed over to the

new user when it is to be reused again. The durability and service life of the product are not limited and may vary. They depend on the conditions of use and the environment. In addition, daily use and the degree of disability can affect the durability and service life of the product. Regular inspection and the performance of all necessary service and maintenance work can positively influence the durability and service life of the product.

## 13. Disposal/environmental compatibility

---

Environmental protection is important. Therefore interco does not work with hazardous substances and tries to integrate environmental aspects into its daily work and thinking and to pass these approaches on to our suppliers if necessary. All employees are frugal in using the resources provided to us.

All packaging materials are recyclable. interco participates in the Dual System so that environment-friendly disposal of transport packaging is ensured. The product can be broken down into its individual material components at the end of its useful life. Dispose of these raw materials, depending on their type, in accordance with the applicable environmental protection regulations.

## 14. Possible faults

---

All serious incidents and faults occurring in connection with the product and its use must be reported immediately to the specialist dealer. Furthermore, you can inform the Compliance Officer of interco and the competent authority in your country.

Your Compliance Officer at interco:  
Ute Markwald (E-mail: [msib@interco-reha.de](mailto:msib@interco-reha.de))

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Eitorf, 2021-01-12

  
Michael Markwald  
Geschäftsführer  
*Chief Executive Officer*

## 16. Warranty policy

In addition to our General Terms and Conditions, we grant a two-year warranty on all products supplied by us. Unusable or defective parts will be repaired or replaced free of charge during the warranty period. The warranty shall exclude design-related modifications.

interco Group does not assume any liability for damages resulting from the combination of our products with third-party products of any kind, unless our company has given its express approval for such a product combination.

Furthermore, warranty also excludes any defects occurring as the result of normal wear, excessive stress, violent and willful damage, as well as inappropriate use. Warranty obligations shall cease to apply if original interco replacement parts are not used for repairs and if repairs and modifications are not performed by interco. The installation of additional parts or changes to the interco products by others than the manufacturer are not permitted and can be subject to legal

action if they become known. Likewise, the warranty obligations of interco Group shall cease to apply in such cases and any liability claims directed at the manufacturer shall no longer be valid.


Certain screws and nuts on your interco product may become loose when the product is used. Re-tighten these at periodic intervals or arrange to have this done by your local dealer or by interco.

### Note!

Expiry of the manufacturer's warranty  
The product must be checked regularly – at least once a year – as part of a (payable) service. Otherwise interco Group cannot provide any warranty.

interco cannot grant warranty if third party components (not manufactured by interco) are used without authorization by interco.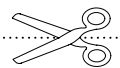

### Warranty extension

On receipt of this form, we will grant an extended warranty for your MAGICLINE® for a total of two years. Please fill out this warranty card carefully and completely and return it to us. Thank you very much.

Funding agency/health insurance fund: \_\_\_\_\_

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
Status as per March 2021 | Subject to technical alterations without prior notification

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Management system certified as per

**DIN EN ISO 13485:2016**

Reg.-Nr. 73 105 6351

TÜV Hessen

**DIN EN ISO 9001:2015**

Reg.-Nr. 73 100 6351

TÜV Hessen

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© interco Group GmbH, Eitorf, 2021