interco[®]
System!

ROOMY NEW EDITION[®]

Universelles Untergestell
Universal undercarriage

Leicht. Flexibel. Absenkbar.
Lightweight. Flexible. Can be lowered.

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Allowing curiosity unlimited space

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ROOMY NEW EDITION and ROOMY are optimum seat shell undercarriages for indoors. Together with a specific seating system, it helps the patient achieve mobility and independence.

Flexibility that integrates

ROOMY NEW EDITION is a movable room undercarriage that is adapted precisely to indoor requirements. Freedom of movement and integration at home or in the kindergarten are no longer a problem with this system. Because the child's seat shell can be lowered almost right down to the floor or raised to table height.

Easy to operate and growing with the patient

ROOMY NEW EDITION distinguishes itself through its low overall weight of approximately 17 kg – with lightweight rollers and without seat shell. This enables comfortable and quick handling for parents and carers. Even more: the construction of the frame offers the possibility of assembling various seating systems with the help of a trapezoid adapter. This enables adaptation to the requirements of different age groups as well as use of the system over various stages of development. ROOMY NEW EDITION and ROOMY can be combined with the functional seat shell systems MAGICLIGHT, MAGICLINE, ORTHOLIGHT and ERGOLINE passive.

ROOMY NEW EDITION (Outdoor-Version)
ROOMY NEW EDITION (outdoor version)

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Variable adjustment and diverse accessories

ROOMY NEW EDITION is made from oval steel tubing and is given permanent protection by a powder-coated, environmentally-friendly, synthetic coating. The four swivelling, lightweight rollers are easy to brake and can be secured in differing directions. The seat unit can be inclined within an adjustment range of 0 to 35 degrees via gas-pressure springs. The maximum load, including seat shell, is 70 kilograms. ROOMY NEW EDITION is available in two versions: with lightweight rollers (125 or 100 mm) or tyres (12" or 16"), air-filled or accident-proof. To enable steered movement, ROOMY NEW EDITION must be equipped with a removable pushing grip.

We recommend that you select the suitable technical aids for your child together with your specialist doctor, therapist and with a specialist medical equipment dealer of your choice. We at interco shall be pleased to provide you with assistance on site. Questions? Simply give us a call – we shall be glad to help.

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In care with seat shells and modular seating systems for children, undercarriages are obligatory for indoors or outdoors. They serve the mobility and transport of the insured party. An indication for undercarriages is therefore always the accommodation of the seat shell.

Your personal, local specialist advisor will assist you in the planning of an individual solution, taking account of your personal requirements. We at interco guarantee the availability of replacement parts or readjustment over a period of years.

ROOMY Untergestell
ROOMY undercarriage

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ROOMY NEW EDITION (Indoor-Version)
in Kombination mit MAGICLIGHT

ROOMY NEW EDITION (indoor version)
in combination with MAGICLIGHT

ROOMY in Kombination mit ORTHOLIGHT
ROOMY in combination with ORTHOLIGHT

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| ROOMY NEW EDITION | with 125-/100-mm light-weight rollers | | with 12" wheels | with 16" wheels |
|--|--|------------------------|---|------------------------|
| | short version | long version | | |
| Maximum length | 800/750 mm | 900/850 mm | approx. 990 mm | approx. 1040 mm |
| Maximum width | 600/580 mm | 600/580 mm | approx. 700 mm | approx. 700 mm |
| Minimum height, upper edge of the adapter | approx. 240/210 mm | approx. 240/210 mm | approx. 315 mm | approx. 315 mm |
| Maximum height, upper edge of the adapter | approx. 680/650 mm | approx. 680/650 mm | approx. 750 mm | approx. 750 mm |
| Total weight without shell | 17/16 kg | 17/16 kg | 23 kg | 24 kg |
| Max. load (including seat shell) | 70 kg | 70 kg | 70 kg | 70 kg |
| Inclination adjustment range in intermediate and upper height adjustment setting | 0 to 35° (-10 to 25°)* | 0 to 35° (-10 to 25°)* | 0 to 35° (-10 to 25°)* | 0 to 35° (-10 to 25°)* |
| Inclination adjustment range in lower height adjustment setting | up to approx. 15° | up to approx. 15° | up to approx. 15° | up to approx. 15° |
| Equipment | Lightweight rollers with brakes and directional parking; Option: easily removable, height and angle-adjustable pushing grip | | 4 air or accident-proof tyres in 12" or 16" wheelsize (rear) and 200×50 mm wheelsize (front); drum brakes with Bowden cable and brake levers on the pushing grip | |

| ROOMY | with 75-mm wheels |
|--|--|
| Weight without shell | approx. 12 kg |
| Max. load (including seat shell) | 60 kg |
| Max. length | 590 mm |
| Max. width | 500 mm |
| Adapter plate angle adjustment | -10° to 30° |
| Max. total height (to the upper edge of the adapter plate) | approx. 500 mm |
| Min. total height (to the upper edge of the adapter plate) | approx. 370 mm |
| Equipment | double lightweight rollers with brakes and parking |

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Status as per January 2021
Subject to technical alterations
without prior notification

Managementsystem zertifiziert nach
Management system certified as per
DIN EN ISO 13485:2016
Reg.-Nr. 73 105 6351
TÜV Hessen
DIN EN ISO 9001:2015
Reg.-Nr. 73 100 6351
TÜV Hessen

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© interco Group GmbH, Eitorf, 2021
interco Group GmbH