CE

SIMPLY®

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English

Contents

1.	Introduction	34
1.1.	Product description	34
1.2.	Product overview	34
1.3.	Installation plan	34
1.4.	Environmental conditions	35
2.	Medical requirements	35
2.1.	Field of application/Target group	35
2.2.	Indications	35
2.3.	Contraindications	35
2.4.	Intended use	35
3.	Product labeling	35
4.	Safety instructions	36
5.	Caution symbols	37
6.	Scope of delivery	39
6.1.	Basic configuration	39
6.2.	Options/Accessories	39
6.2.1.	Basic configuration	39
6.2.2.	Lower extremities	39
6.2.3.	Upper extremities	40
6.2.4.	Other Accessories	40
7.	Settings/Operation	40
7.1.	Basic configuration	40
7.1.1.	Trapezoidal adaptor plate	40
7.1.2.	Adjusting the inclination angle of the seat shell	41
7.1.3.	Settings and adjustment options for the push bar	42
7.1.4.	Folding mechanism for transporting SIMPLY®	43
7.1.5.	Swiveling the support rollers	43
7.1.6.	Dual rear-wheel drum brakes	44
7.1.7.	Checking the tyre pressure	44
7.2.	Options/Accessories	45
7.2.1.	Basic configuration	45
7.2.2.	Lower extremities	45
7.2.3.	Upper extremities	46
7.2.4.	Other Accessories	46
8.	Compatibility/Combination options	47
9.	Technical data	48
9.1.	Overall dimensions of SIMPLY®	48
9.2.	Tyre pressure table	48
10.	Transport/transport lock/climbing stairs	49
10.1.	General information on transport	49
10.2.	Transport with an AMF force application point system	49
10.3.	Transport on aircraft	49
10.4.	Climbing stairs	49
11.	Maintenance plan	49
11.1.	Cleaning	49
11.1.1.	Materials used	49
11.1.2.	Cleaning of materials	50
11.1.3.	Disinfection of the product	50
11.2.	Maintenance	50
11.3.	Service	51
11.3.1.	Seating system	51
12.	Reuse/durability	52
13.	Disposal/environmental compatibility	52
14.	Possible faults	52
15.	EU Declaration of Conformity	53
16.	Warranty policy	57

Information

Date of last update:
September 2022

Revision status:
SIMPLY_Gebrauchsanweisung_interco_de-en_02

Please read these operating instructions carefully! Observe all safety instructions. If you have any questions, please contact your specialist dealer or interco.

Note

As a rule, all interco products consist of a seat shell and/or an undercarriage. To simplify matters, only the interco product is mentioned in the following.

In these operating instructions we use the masculine form exclusively for the male/female users of an interco product and its male/female operators (parents, male/female caregivers, male/female therapists).

This way we wish to maintain the readability and comprehensibility of the texts. At the same time, we would like to expressly point out that this naturally includes the female form and other gender identities.

Foreword

Dear Customer,

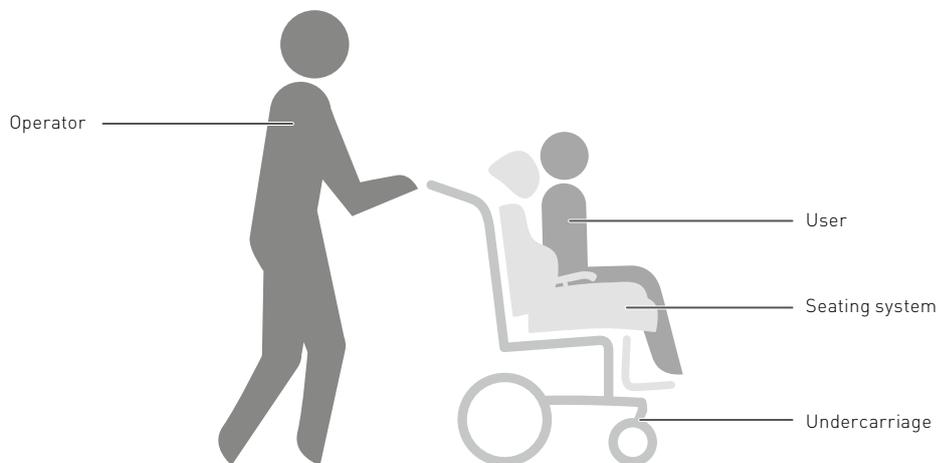
Thank you very much for choosing a quality product from interco. This is intended to contribute to a sustainable improvement in the everyday life and quality of life of the user. To ensure that you can always handle

the product safely and use it for a long time, we have described what you should bear in mind during use in these operating instructions.

It is important that you carefully read and understand all sections that apply to your interco product. Keep the operating instructions for future reference. Please follow all the safety instructions and symbols to prevent any type of injury. If you should have any questions about your product and its use, please ask your local qualified medical supplies dealer or contact us directly. We would also be pleased to inform you about further customization and adaptation options for your interco product.

Please observe the following in particular:

- ▶ Parents, accompanying persons, nurses and family members who operate this product should be instructed in the use of the product by authorized specialist personnel using these operating instructions.
- ▶ Your interco product was customized to the needs of the user. Subsequent changes to the product may only be made by trained specialist personnel. We recommend regular checks of the product and the product adaptation to ensure optimal long-term operation.
- ▶ Your interco product is only approved for combination with one of the products listed later on. interco will not accept any liability for combinations with medical devices and/or accessories from other manufacturers.
- ▶ Technical modifications may only be made with the written consent of interco.
- ▶ interco will not accept liability for modifications, changes to parts of the frame of the product and unapproved repairs to this product.
- ▶ interco reserves the right to technical changes to the model described in these operating instructions.

1. Introduction

1.1. Product description

SIMPLY® is the street chassis for the interco seat unit range and offers a variety of adjustment possibilities. The inclination of the seat unit can be continuously adjusted via gas spring. Pneumatic tyres and drum brakes supplied as standard ensure a high degree of safety and comfort. At the front, SIMPLY® features

swivel wheels so that it is extremely manoeuvrable and particularly easy to push. SIMPLY® can be folded up easily and quickly for transportation and it fits into any car boot. The SIMPLY® street chassis is suitable for all interco seat units featuring a trapezoidal adaptor. SIMPLY® has been especially designed for outdoor use. For indoor use, we would recommend the ROOMY® NEW EDITION home chassis.

1.2. Product overview

- 1 Front wheels
- 2 Rear wheels
- 3 Lower frame
- 4 Upper frame
- 5 Trapezoidal seat unit adaptor
- 6 Back
- 7 Handle

1.3. Installation plan

Authorization for delivery

SIMPLY® may only be delivered to the end customer by authorized specialist dealers, medical device consultants or by the medical device consultants of interco Group.

Scope of delivery

SIMPLY®'s basic features are:

- ▶ Trapezoidal seat unit adaptor
- ▶ Seat inclination via gas spring 300N (from a seat width of 45 cm with two gas springs)
- ▶ Continuous back reclining angle adjustment up to 30°
- ▶ Notched back reclining angle adjustment up to 30°
- ▶ Handle adjustable in height and angle
- ▶ Depth adjustment of the seat unit adaptor support
- ▶ Drive wheels optionally 12", 16", 22" or 24" with quick release axle and drum brake
- ▶ Integrated wheelbase extension
- ▶ Support rollers
- ▶ Parking brake handles

The trapezoidal adaptor plate is standardly mounted to all interco chassis'. Therefore, it is possible to attach an interco seat unit featuring a trapezoidal adaptor to various interco chassis'. The installed trapezoidal adaptor plate basically consists of three functional components:

1. Supporting plate with guide bushings to receive the adaptor.
2. Locking pin for securing the seat unit against sliding out during assembly.
3. Two wing screws (1 on the right and 1 on the left) for the assembly and attachment of the footrests (15 x 15 mm) (see section 7.2.2.)

Setup/assembly

The product is delivered fully pre-assembled to the end customer, thus no components need to be assembled on-site. If options are installed at a later date, the assembly instructions are enclosed with the option.

The medical product advisor of the specialist dealer must ensure that the instructions are carried out correctly. (see below) The medical device consultant of interco Group does not have to be present during the installation of the product. However, he can be consulted for support in case of questions.

Note!

Please make sure that the individual adjustment of the adaptor system has been carried out correctly. You can do so by checking that the adaptor rests flat on the front edge of the adaptor plate and the locking pin is fully engaged.

Instruction of the customer

All functions of SIMPLY® which are explained in the operating instructions must be demonstrated to the customer. The operating instructions remain with the customer. All adjustments to the product that directly affect the user may only be carried out by medical device consultants or in consultation with these.

1.4. Environmental conditions

The SIMPLY® medical device must always be stored in a dry place to prevent possible corrosion. Please keep to ambient temperatures of -10°C to +40°C. Sand or other dirt particles should not be allowed to penetrate the product. Use in salty water is not permissible.

2. Medical requirements

2.1. Field of application/Target group

If there is an indication for a seat shell

- ▶ To accommodate the existing seat shell

2.2. Indications

Seating aids serve to compensate for a disability with moderate to severe bad posture and/or Postural instability. The underlying Damage as a cause of impairment of Sitting are diverse. Mostly the interaction of Pelvis and spine either by congenital or acquired damage to the trunk (bony or joint structures) or by congenital or acquired functional disorders of the musculoskeletal system (muscular, neuronal, cerebral, injury-related) disturbed. As exemplary clinical pictures/diseases can be named: cerebral movement disorders, Myelodysplasia, muscular dystrophy or atrophy, multiple sclerosis, other diseases with scoliotic spinal deformities.

2.3. Contraindications

Individuals, especially children, who, because of their age or other physical limitations not yet or generally

not take a sitting position may not be positioned in a seat pan unless this is expressly done by one doctor prescribed. Any kind of pain when ingested a seated position can be a contraindication represent. In general, the indication should Seat shell, of a seating system always medically and/or be accompanied therapeutically and be indexed by regulation.

In addition, no other contraindications known.

2.4. Intended use

When supplying individual seat shells and modular seating systems for people with disabilities mobility (severely limited ability to walk up to walking disability) are chassis or bases mandatory for indoor and/or outdoor use. The chassis and undercarriages take individual Seat shells and modular seating systems and enable the mobility and transport of patients including transport in motor vehicles if this is explicit allowed.

3. Product labeling

Example: Product labeling SIMPLY® SB25 (Art. no. 74600-35)

- A Type designation
- B Manufacturer's data
- C Year of manufacture
- D Serial number (always indicate in case of complaints and other questions)
- E Article number
- F UDI Code as barcode to scan
- G UDI Code in human readable form
- H Observe warnings/safety instructions
- I Read operating instructions prior to use
- J Maximum weight

The serial number and the size of the undercarriage and/or the UDI-DI of SIMPLY® must always be stated when ordering spare parts or accessories.

The serial number with all details is attached to the lower left part of the frame (viewed in the direction of travel) on the inside.

4. Safety instructions

Please note that you are responsible for the safety of the user. This could be at risk if you do not follow the safety guidelines. Nevertheless, it is not possible to cover all potential circumstances and unforeseeable situations in these guidelines. Therefore common sense, caution and prudence are required when handling the product. Every person operating the product should know and understand all the instructions. Furthermore, you should explain the operation of the product and its accessories to every person using the product.

Please contact your local specialist dealer or interco directly if the instructions are unclear and a more detailed explanation is required or if you have any other questions.

Meaning of the safety instructions

Warning!

Warning against possible severe damage due to accidents or injuries.

Caution!

Warning against possible damage due to accidents or injuries.

Note!

Warning for possible technical damage.

Product-specific safety instructions

Warning!

Risk of injury from unattended parking

If the product is left unattended with user, the user or other persons may be injured.

- ▶ Please make absolutely sure that the person sitting in the product is properly secured at all times by a suitable restraint system (harness).
- ▶ Never leave the user sitting in the product unattended, not even when the user has been securely strapped in and the brakes have been set.
- ▶ Never leave the product alone with the user without expert supervision to exclude the risk of injury to the user or other persons.

Warning!

Risk of suffocation due to the packaging material

Incorrect handling of packaging material may cause a risk of suffocation.

- ▶ Keep the packaging materials out of reach of children.

Warning!

Risks when climbing stairs

When using stairs, there may be a risk of injury to the user of the product or the product may be damaged. Therefore only lift the product without the user sitting in it.

- ▶ Stairs are only permitted to be negotiated with

the assistance of accompanying persons.

- ▶ If facilities such as access ramps or lifts are available for this purpose, these must be used. In the absence of such facilities, two accompanying persons can carry the product over obstacles without the user.
- ▶ Techniques for climbing stairs are explained in detail in Section 10; it is imperative that you comply with these.
- ▶ Make sure that the accompanying persons only grip the product on firmly mounted or bonded components.
- ▶ Do not lift the product by the foot rest, head rest, arm rests or similar parts.

Warning!

Risks of self-initiated modification of the settings

If settings are changed autonomously, the function of the product and the safety of the user may be affected.

- ▶ Adhere to the settings made by the specialist dealer. You may only adjust settings by yourself as described in Section 7.
- ▶ If you experience problems with settings, please contact your specialist consultant or interco directly.
- ▶ Discuss all changes to the settings properly with the specialist personnel or therapist.
- ▶ It is not permissible for any parties other than interco to make additions or modifications to the product and any cases that should become known can be subject to legal action. In addition, the warranty obligations of interco Group shall cease to apply in such cases and any liability claims directed at the manufacturer shall no longer be valid.

Caution!

Risk of injury due to operating errors or lacking knowledge of the operating instructions

If the instructions in the operating instructions are disregarded, application errors may occur and the safety of the user may be impaired. Please read these operating instructions carefully first.

- ▶ Familiarize yourself with the functions of the product and its handling prior to initial use. Practice handling the product before use.
- ▶ Observe the maximum load capacity of the product (see Section 9: Technical Data).
- ▶ Do not use the foot rest when getting into or out of the product.

Caution!

Risk of tipping due to incorrect center of gravity / shift in center of gravity

If the product's center of gravity is shifted significantly, for example on slopes and inclines or if the user leans out of the product too far, the product may tip or roll over.

- ▶ Test the effects of shifts in the center of gravity on the behavior of the product, for example, on declines, inclines, and lateral slopes or while negotiating obstacles only with the reliable support of an assistant.
- ▶ Always stand the product on horizontal, flat surfaces only. If it is unavoidable to park it on an incline, put the seat in an upright position. There is a risk of the product tipping backwards on inclines in the lying position.
- ▶ When grasping objects (lying in front of, to the side or behind the device) make sure that the user does not lean out of the seat too far.
- ▶ Do not hang heavy shopping bags or similar on the push bar or the levers.

Caution!

Risks in public road traffic

Dangerous situations can occur when driving on public roads.

- ▶ Please note that you must observe traffic regulations when traveling on public streets.
- ▶ In dark conditions wear bright clothing wherever possible in order to be more visible. We also recommend installing an active light source.

Caution!

Risk of injury to hands and fingers

Your hands or fingers can get caught during adjustment and assembly work.

- ▶ Always be aware of your fingers during operation and all adjustment and assembly work.

Caution!

Dangers due to loosened components during use

Screws and nuts on the product may become loose when the product is used. This can result in risks of accidents and injuries for the user and the operators.

- ▶ Check all screws and nuts regularly and retighten them.
- ▶ Alternatively, you can also instruct your specialist dealer or interco to do this.

Caution!

Dangers when making adjustments to the product and connections

If you do not carry out adjustments to the product properly, the stability and safety of the product may be affected.

- ▶ Ensure that the screws are tightened again after all adjustment and assembly work.
- ▶ Always make sure that the seating system is securely connected to the undercarriage.
- ▶ If you have a question regarding the connection of the seating system and the undercarriage, always contact your specialist dealer or interco directly.
- ▶ Carry out a regular visual inspection of all connections of the product. Consult your local specialist dealer in case of doubt.

Caution!

General risks due to careless handling of the product

The product is only approved for its intended use. Improper or careless use can result in safety and injury risks for the user and other persons.

- ▶ Avoid actions, such as driving against an obstacle (stair, curb) without using the brake or down from elevated surfaces.
- ▶ Use caution when handling fire (e.g. a burning cigarette)! Upholstery and coverings could catch fire.
- ▶ Make sure to keep children from playing with the therapy chair or any parts thereof. There is a risk of injury due to trapping and crushing hands and fingers.
- ▶ Children must not play next to, underneath or in front of the seating system when the seating system is in motion and/or the seat or back unit is in use.

5. Caution symbols

Note: All screw connections, that were loosened by changing the settings, must be re-tightened!

Place of mounting: upper frame on the side rails

Note: This wheelchair is completed with a special wheelchair restraining system. It is allowed to use this wheelchair as seat in vehicles, if the security belts are used according to the specifications.

Place of mounting: lower frame, on top of the crossbar

Note: For adjusting the angle, open control lever at least 3 turns!
Place of mounting: push handle on the right on top

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Note: Assure appropriate tilting protection when adjusting seat angle!
Place of mounting: push handle on the left on top

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Note: Check the catch-mechanism and secure with locking pin after attaching seat!
Place of mounting: on the front of the locking cams, in the middle

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Check the catch-mechanism and secure with locking pin after attaching seat!

Note: It is not permitted to use this wheelchair as a car seat.
Place of mounting: lower frame, on top of the crossbar

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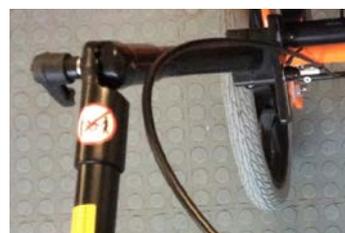
Note: Do not use on stairs!
Place of mounting: push handle on the right on top

Note: Do not carry!
Place of mounting: push handle on the left on top

6. Scope of delivery

6.1. Basic configuration

SIMPLY® is a functional undercarriage with

- ▶ with 12"/16"/22"/24" wheels with pneumatic tires or solid tires (no additional charge)
- ▶ drum brakes, parking brake handles
- ▶ includes trapezoidal mounting device for seat shell
- ▶ seat shell adapter with depth adjustment
- ▶ tilt-in-space mechanism using one gas-pressure spring (SB 35 and SB 40) or tilt-in-space mechanism using two gas-pressure springs (SB 40 and SB 45) by approx. 35°
- ▶ back angle adjustment, manual
- ▶ height- and angle-adjustable push bar
- ▶ pivoting support rollers

6.2. Options/Accessories

6.2.1. Basic configuration

Art. no.	Article	Size
40110	Trapezoidal adapter for mounting under the seat shell	
73030	Stabilizer bar for SIMPLY® top frame, able to be fixed in position	
73014	Support rollers	
73009	Knee lever brake for SIMPLY®	
73090-35	Continuous back angle adjustment using gas-pressure springs for SIMPLY® with 35-cm seat width in combination with ERGOLINE® (optional combinations with art. no. 40211 + 40213 or 41041 + 40110)	
73090-40/45	Continuous back angle adjustment using gas-pressure springs for SIMPLY® with 40-cm or 45-cm seat width in combination with ERGOLINE® (optional combinations with art. no. 40211 + 40213 or 41041 + 40110)	
73005	Distance block/ widened wheel base for SIMPLY® front wheels	
73007	Support rollers, able to be swiveled out of the way using one's foot, for SIMPLY®	

6.2.2. Lower extremities

Art. no.	Article	Size
73012	Leg rest system on center column, height- and angle-adjustable, foldable up and back	size 0-3
46013	ALULINE leg rest system with incrementally adjustable holder for leg rest system Foot rest, complete version, foot rest powder coated in black, includes corner guards and foot rest overlay for all interco undercarriages	size 0-3
46016	ALULINE leg rest system with incrementally adjustable holder for leg rest system Foot rest with individual foot supports, foot rest powder coated in black, includes corner guards and foot rest overlay for all interco undercarriages	size 0-3
46010	Leg rest, complete version, with fixed foot brackets and foot guide slots	size 1-3
46012	Leg rest, complete version, with foot bracket with continuous angle adjustment and foot guide slots	size 1-3
46014	Leg rest with individual foot supports, with foot bracket with continuous angle adjustment and foot guide slots	size 0-3
46015	Leg rest, with individual foot supports and fixed foot brackets and foot guide slots	size 0-3
46020	Holder for leg rest system in fixed 90° version or fixed 110° version	size 1-3
46021	Holder for leg rest system with continuous angle adjustment	size 1-3
46030	Calf support, complete version	
46031	Black synthetic leather cover for calf support, complete version	
46120	Calf support, single version	
46121	Black synthetic leather cover for calf support, single version	
46530	Plastic foot rest riser, black	

Art. no.	Article	Size
46551	Heel stop for leg rest system, complete version	
46552	Heel stop for leg rest with individual foot supports	
46510	Heel stop cushion without cover	
46511	Black synthetic leather cover for heel stop cushion	
46520	Foot rest cushion without cover	
46521	Black synthetic leather cover for foot rest cushion	
46410	Foot straps with hook and loop fastener with padded instep	
46411	Foot straps made of leather with padded instep	
46412	Foot straps with deflection roller and padded instep with hook and loop fastener	
46419	Foot straps with incremental adjustment and padded instep	
46420	Foot straps with clip buckle and padded instep	
46210	Foot shells made of aluminum	size 1-3
46414	Foot gaiters without forefoot restraint made of neoprene	size 1-3
46416	Foot gaiters with forefoot restraint made of neoprene	size 1-3

6.2.3. Upper extremities

No options available

6.2.4. Other Accessories

Art. no.	Article	Size
73004	Quick release axles for front wheels for SIMPLY®	
73035	Stop for front wheels	
73001-12	Wheel protectors, includes mounting materials for SIMPLY® 12", for SIMPLY® 16",	
73001-16	for SIMPLY® 22" and for SIMPLY® 24"	
73001-22		
73001-24		
73002-12	Spoke protectors, transparent for SIMPLY® 12", for SIMPLY® 16",	
73002-16	for SIMPLY® 22" and for SIMPLY® 24"	
73002-22		
73002-24		

7. Settings/Operation

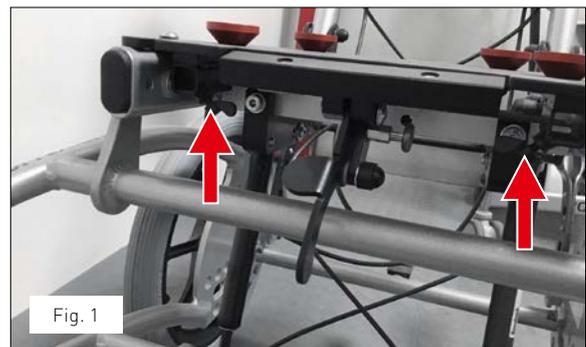
Every interco product is customized to the individual needs of the user and configured accordingly. To do justice to all possible product designs, the list of options and accessories as well as their descriptions is therefore somewhat more extensive. Therefore please bear in mind that this Section may also describe components that possibly do not exist on your product.

7.1. Basic configuration

7.1.1. Trapezoidal adaptor plate

The trapezoidal adaptor plate is standardly mounted to all interco undercarriages. Therefore, it is possible to attach an interco seat unit featuring a trapezoidal adaptor to various interco undercarriages. The installed trapezoidal adaptor plate basically consists of three functional components:

1. Supporting plate with guide bushings to receive the adaptor.
2. Locking pin for securing the seat unit against sliding out during assembly.
3. Two wing screws (one on the right and one on the left – Fig. 1) for the assembly and attachment of the footrests (see section 7.2.2.).

Note!

Please make sure that the individual adjustment of the adaptor system has been carried out correctly. You can do so by checking that the adaptor rests flat on the front edge of the adaptor plate and the locking pin is fully engaged.

Attachment of the seat unit

The seat unit is fixed in the trapezoid adapter plate (Fig. 2) with the help of the trapezoid adapter. Carry out the following steps and check the safe halt of the seat unit afterwards.

Fig. 2

Fig. 3

Fig. 4

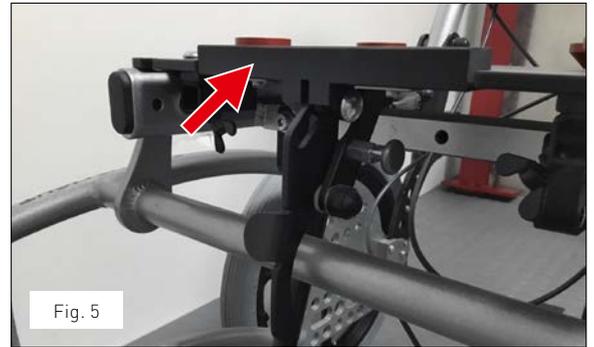
Fig. 5

Fig. 6

Fig. 7

Unlock the seat shell fixation by pulling the locking pin and turning it 90° (Fig. 3).

Position the seat unit with the rear part of the trapezoidal adaptor on the trapezoidal plate (Fig. 4) and push the seat unit (and thereby the trapezoidal adaptor) backwards into the trapezoid plate until the locking pin engages audibly (Fig. 5). Lock the seat shell fixation again by turning the locking pin once more by 90° (Fig. 3). At last, check that the seat shell and the lock are firmly fixed.

When attaching the seat shell with a locking pin, first check whether the pin is locked. Check whether the release button is locked in. If yes, the locking pin should reach out on top of the adaptor plate. If not, turn the locking pin until it locks. Position the seat unit with the rear part of the trapezoidal adaptor on the trapezoidal plate. Push the seat unit (and thereby the trapezoidal adaptor) backwards into the trapezoid plate until the locking pin engages audibly.

Caution!

Take care to always make sure that the seat unit is closely attached to the chassis and secured by the locking unit. Take care to always make sure that the locking of the trapezoidal adaptor is engaged and that the seat unit cannot be removed from the chassis.

Removal of the seat unit

Pull the locking pin downward and turn by 90° (Fig. 6). Open the seat shell fixation by pushing down the lever (Fig. 7). Now you can pull the seat shell out towards the front. To remove the seat shell pull the locking pin downward and turn by 90°. This way the locking pin will remain in its unlocked position. Open the seat shell fixation by pushing down the lever. Now you can pull the seat shell out towards the front. Remove the seat shell from the undercarriage. Turn the locking pin until it locks audibly.

7.1.2. Adjusting the inclination angle of the seat shell

The SIMPLY® is equipped with gas-pressure spring adjustment for inclination of the seat shell. To adjust the inclination angle of the seat shell, proceed as follows:

- ▶ Activate the parking brakes (see section 7.1.6.).
- ▶ Pull both gas-pressure spring release levers mounted on the push bar frame (Fig. 8).
- ▶ Adjust the inclination angle of the seat shell by pulling the push bar up or pressing it down while pulling both gas-pressure spring release levers.
- ▶ Let go of both release levers to lock the adjusted inclination angle of the seat.
- ▶ Release the parking brakes, as appropriate (see section 7.1.6.).

Caution!

Never pull the release levers without a firm grip on the push bar if a patient is seated in the wheelchair. Prior to inclination of the seat shell, ensure that the support rollers (if provided) are folded down and engaged.

7.1.3. Settings and adjustment options for the push bar

The push bar supports various adjustment options. This enables optimal adjustment to the person pushing the chair.

The following adjustments are possible:

- ▶ Adjusting the angle of the top piece of the push bar
- ▶ Adjusting the height of the push bar unit
- ▶ Adjusting the inclination angle of the push bar unit relative to the vertical

Adjusting the angle of the top piece of the push bar

To adjust the angle of the push bar, proceed as follows:

- ▶ Activate the parking brakes (see Section 7.1.6.).
- ▶ Loosen the screw connections on the right and left ratchet joints on the push bar by turning the clamp levers (Fig. 9) counterclockwise until the top piece of the push bar can be moved.
- ▶ Move the push bar into the desired position.
- ▶ Hold the push bar in the desired position with one hand and secure it in place by tightening the corresponding clamp lever with the other hand.
- ▶ Secure the screw connection on the other side as well by tightening the clamp lever.

Caution!

When tightening the ratchet joints make sure that the teeth engage properly.

- ▶ Release the parking brakes, as appropriate (see section 7.1.6.)

Adjusting the height of the push bar unit

Adjustment of the height of the push bar is always necessary when adapting the SIMPLY® to suit the person pushing the chair or if the inclination angle of the seat shell has been changed.

To adjust the height of the push bar, proceed as follows:

- ▶ First, loosen the quick-release fasteners on the fork tubes by opening the two quick-release levers (Fig. 10) outwards.
- ▶ Then adjust the push bar to the desired height. To do this, pull or press the push bar to the desired height.
- ▶ Secure in position by tightening the quick-release connectors. To do so, tilt the lever on the quick-release fasteners back in.
- ▶ If the clamping pressure is not sufficient (push bar not fixed securely), turn the screw across from the quick-release fastener clamp lever clockwise (with the lever in the open position), until sufficient clamping pressure is achieved by closing the clamp lever.
- ▶ Repeat this process, as necessary.

Adjusting the inclination angle of the back unit, including the push bar, relative to the vertical

In addition to the adjustment options already described above, it is also possible to swivel the entire push bar back by up to 30°. This can become necessary, for ex-

Fig. 8

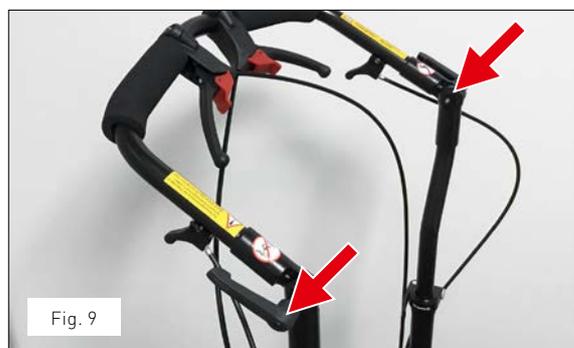
Fig. 9

Fig. 10

ample, if a seat shell is being used with a seat angle of more than 90° up to max. 120°. This enables the gap between the back of the seat shell and the push bar to be as uniformly small as possible. This adjustment option is not intended for individual adjustment to the person pushing the chair.

To adjust the inclination angle of the push bar, proceed as follows:

- ▶ Loosen the two nuts located in front of the pivot point (Fig. 11) for the push bar folding mechanism by turning them counterclockwise.
- ▶ Apply axial pressure to the nuts in order to press the carriage bolts out of the guide on the ratcheting element until it is possible to freely adjust the inclination angle.

- ▶ Move the push bar into the desired position.

Note!

Apply pressure to the rounded head in order to press the square head of the screw into the ratcheting element at the required inclination angle.

- ▶ Check that the carriage bolt is correctly seated in the ratcheting element.
- ▶ Secure in this position by turning both nuts clockwise to tighten them.

Note!

It is also possible to maintain the inclination angle of the back unit, including the push bar unit, when the push bar is folded down (e.g. during transport).

7.1.4. Folding mechanism for transporting SIMPLY®

SIMPLY® is equipped with a folding mechanism to make transportation easier. The back unit including the handle is folded forwards and the rear wheels can also be removed. Thereby, extremely small overall transportation dimensions are achieved. If the dimensions have to be further reduced for transportation, removable swivel front wheels are available as a special accessory.

Caution!

Take care to always make sure that the anti-collapse locks are engaged if the SIMPLY® is used so that it may not fold up unintentionally.

Folding SIMPLY®

When folding SIMPLY® for transportation, proceed as follows:

- ▶ Remove the seat unit (see section 7.1.1.).
- ▶ Loosen the height adjustment of the handle (see section 7.1.3.).
- ▶ Loosen the screw connections for the handle angle adjustment on the left and right sides of the handle (see section 7.1.3.).
- ▶ Loosen the catch for the handle folding mechanism by pressing down the central release bar connecting the two locking plates on the lower ends of the two handle upright tubes with your foot while simultaneously swinging the handle forwards.
- ▶ The back unit including the handle can now be folded forwards until it rests against the SIMPLY® frame.
- ▶ Next, remove the rear wheels. In order to do this, lift the SIMPLY® frame slightly so that the respective rear wheel is raised off the ground.
- ▶ Take hold of the rear wheel spokes and press the button located at the centre of the wheel axle with your thumb.
- ▶ Hold the button pressed and remove the wheel from the axle mounting. The wheel brake must be released during this operation.
- ▶ Repeat the operation for the second rear wheel.
- ▶ If necessary, fold up the support rollers as described in section 7.1.5.
- ▶ Depending on the version, remove the swivel wheels as described in section 7.1.4.
- ▶ Now, you can put SIMPLY® into the vehicle.

Fig. 11

In order to load the folded SIMPLY®, grasp the crossbar at the back as well as the connecting tube in the front in the middle and lift the undercarriage.

Caution!

Do not use the folded push bar to lift the undercarriage!

Unfolding SIMPLY®

When unfolding SIMPLY®, proceed as follows:

- ▶ First of all, check that the SIMPLY® parking brakes are released.
- ▶ Lift SIMPLY® at the rear.
- ▶ Take hold of the spokes of one rear wheel and press the button located on the axle with your thumb.
- ▶ Hold the button pressed and insert the axle into the axle mounting on the frame.
- ▶ Let go of the button and check whether the wheel is fastened securely. It should not be possible to pull it off!

Caution!

During this check, the button must not be pressed.

- ▶ Repeat this operation for the other rear wheel.
- ▶ Depending on the version, attach the swivel wheels as described in section 7.2.4.
- ▶ If necessary, turn the support rollers as described in section 7.1.5.
- ▶ Fold the handle unit back until it engages audibly.
- ▶ Next, fasten the loose upper part of the handle in position as described in section 7.1.3.
- ▶ Adjust the desired height of the handle; first lift the handle to the desired height.
- ▶ Fasten into position by tightening the quick release connectors as described in section 7.1.3.
- ▶ If clamping is insufficient (the handle can still be moved) proceed as described in section 7.1.3.
- ▶ Attach the seat unit (see section 7.1.1.).

7.1.5. Swiveling the support rollers

The rear end of the frame is equipped with two support rollers.

Caution!

In the normal traveling position, the support rollers must always be folded down and secured.

It may be necessary to swivel the support rollers upwards when negotiating high curbs and for transport (reduces the size of the wheelchair for transport). To do so, proceed as follows:

- ▶ Activate the parking brakes (see section 7.1.6.).
- ▶ Position yourself behind the ECONOMY®.
- ▶ Pull the spring-loaded catch outwards, then swivel the support rollers 90° upwards until the small wheel faces upwards.
- ▶ The spring-loaded catch locks automatically in this position.
- ▶ Release the parking brake as described in section 7.1.6.
- ▶ Follow the same procedure for swiveling back into the original position.

Note!

Always make sure that both support rollers are pointing in the same direction. That is the only way how an effective stabilisation can be ensured!

Support rollers swivel away by foot pressure
Should you wish to extend the support rollers, please also make sure that the adjustment is identical on the left and right sides.

7.1.6. Dual rear-wheel drum brakes

SIMPLY® is equipped with two independently operating drum parking brakes. These act on the two rear wheels. The brake levers are equipped with a three-notch system enabling three different braking levels.

Applying the parking brake

In order to apply the parking brake, proceed as follows:

- ▶ Squeeze both brake levers on the handle (Fig. 12). Ensure that the brake handle does not return to its initial position (the lock engages audibly); only then, the parking brake is correctly applied.
- ▶ Check the braking effect by trying to push the SIMPLY®.

Caution!

If the braking action is insufficient, have the brake adjusted by a specialist workshop or adjust the brake yourself as described below.

- ▶ You can also use the parking brake to make braking easier when going downhill. In this case, pull the brake to the first or second notch position.

Releasing the parking brake

To release the parking brake, proceed as follows:

- ▶ Release the lock by operating the notch lever and allow the brake lever to slide into its original position. In some cases, tension on the notch lever must be relieved by slightly pulling the brake lever before it is possible to release the brake.

Adjusting the parking brake

Tool: Size 8 spanner

Prerequisite: Insufficient braking power of the parking brake: SIMPLY® can be pushed even though the parking brake is applied.

To adjust the parking brake, proceed as follows:

- ▶ Make sure that the parking brake is not applied; if necessary, release the parking brake (see section 7.1.6.).

Fig. 12

- ▶ The Bowden cables of the parking brakes are routed through an adjusting bolt before the brake backing plates, which allows an adjustment of the braking power.
- ▶ By turning this bolt anti-clockwise, the brake cable is brought under greater tension, pre-tensioning the brake. The adjusting bolt is secured by a nut.
- ▶ Before adjusting the brake, release the securing nut by turning it to the left (depending on the version, a size 6 spanner may be required).
- ▶ Next, turn the adjusting bolt anti-clockwise until it becomes more difficult to turn the wheel. To check, lift the SIMPLY® on one side and turn the respective wheel with your other hand.
- ▶ Then, turn the adjusting bolt clockwise until the wheel which has to be adjusted runs freely again.
- ▶ Turn the counter nut located on the thread of the adjusting nut clockwise as far as it will go.
- ▶ If necessary, repeat the procedure for the other wheel.

Advice!

In order to check the adjustment of the brake, pull the brake lever several times before finally fastening the counter nut. If necessary, you have to repeat this operation, as some experience is required to carry out the adjustment.

7.1.7. Checking the tyre pressure

SIMPLY® features pneumatic tyres as standard. In order to keep rolling resistance to an absolute minimum, you should check the tyre pressure at regular intervals. The tyres are standardly equipped with automobile valves, so you can check the pressure by using the tyre gauge at your local petrol station. The required filling pressure is printed on the tyres and is also specified in the "technical data" section of these instructions. To check the tyre pressure, please proceed as follows:

- ▶ Apply the parking brake (see section 7.1.6.).
- ▶ Unscrew the valve cap from the valve by turning it anti-clockwise.
- ▶ Press the tyre gauge nozzle firmly onto the valve.
- ▶ Correct the tyre pressure to the specified value.
- ▶ Remove the tyre gauge nozzle from the valve.
- ▶ Screw the valve cap onto the thread of the valve (clockwise).
- ▶ Repeat the operation until you have checked all tyres.
- ▶ If necessary, release the parking brake (see section 7.1.6.).

Caution!

Depending on the version, the front and rear wheels require different tyre pressures.

7.2. Options/Accessories

7.2.1. Basic configuration

Stabiliser

The stabiliser serves to fix the desired position of the seat and, thereby, to guarantee more stability. By releasing the two lower clamping levers, the stabilisers are unblocked. Now, the two small release handles on the sliding rod can be operated and the seat tilt adjustment can be carried out. When retightening the lower clamping levers, the adjusted inclination is additionally secured.

Spacer block for front wheels (wheelbase extension)

The spacer blocks for the front wheels are pre-installed at the factory when the wheelchair is ordered. No subsequent adjustments are possible. If an installation of spacer blocks becomes necessary at a later date, please contact your local supplier.

Knee lever brake (brake activated by pressure)

Sometimes it is necessary for a wheelchair to possess additional braking action to cope with eventual difficulties as, for example, ground conditions (on slopes or when self-propelled). Knee lever brakes are used in addition to the parking brake. When ordered, they will be assembled and adjusted at the factory. In case the braking action is reduced by distance – alteration (wheel-knee lever brake), this can be regulated by adjusting the sliding nut in the C-rail.

7.2.2. Lower extremities

Footplate

To attach the footplate, proceed as follows:

- ▶ Loosen both wing screws on the right and left sides of the front (long) side of the trapezoidal adapter plate.
- ▶ Loosen both wing screws on the rear edge of the footplate. Ensure that the thread of the wing screws does not protrude onto the guides.
- ▶ Slide the footplate onto the ends of the footrest and fasten at the desired height with the wing screws.
- ▶ Fasten footrests with the wing screws at the trapezoidal adapter plate to the required depth.

Caution!

Do not use the footplate to get in or out of a seating system. The footplate should be folded down or removed beforehand.

Middle footplate

The middle footplate is an alternative to the footrest with footplate. When ordered, it will already be assembled at the factory. Subsequent individual adjustments are possible.

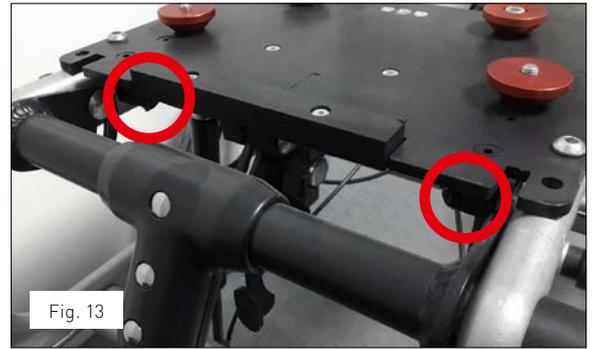
Fig. 13

The middle footplate features various adjustment possibilities to guarantee an optimal adaptation to the occupant's needs:

a) Vertical inclination adjustment

To adjust the middle footplate you have to take the following steps:

- ▶ Apply the parking brake (see section 7.1.6.).
- ▶ Loosen the middle footplate's screw connections located on the right and left of the upper frame by turning the clamping lever anti-clockwise until the upper part can be moved.
- ▶ Bring the middle footplate to the desired position.
- ▶ Hold the middle footplate in the desired position with one hand and fasten it with the other hand by engaging the respective clamping lever.
- ▶ Then, fasten the screw connection on the other side by turning the clamping lever until tight.
- ▶ If necessary, release the parking brakes (see section 7.1.6.)

b) Can be folded backwards

The footrest is folded up by hand.

c) Height and depth adjustment

Loosen the screw at the clamping block of the footrest and adjust the footrest on the middle tube of the middle footplate as required. When finished with the adjustments, tighten the screws again.

Footrest

To attach the footrest, proceed as follows:

- ▶ Loosen the two wing screws on the lower right and left sides of the front (long) side of the trapezoidal adapter plate (Fig. 13).
- ▶ Slide in the footrest to the desired depth.
- ▶ Tighten the two wing screws.

Calf support

Mount the calf support to the center column or the knee brackets using the mounting material supplied. The same screws can be used to adjust the height of the calf support so that the calves rest well.

Heel edge

Mount the additional heel edges to the additional heel edges of the existing footstool using the assembly material supplied.

Fixation

The assembly instructions for the fixation are enclosed.

7.2.3. Upper extremities
No options available

7.2.4. Other Accessories

Wheel guards

The wheel guards cover the upper part of the tyres. They consist of a plastic wing which is bolted to an aluminium mounting plate. To attach this to the chassis, insert the free end of the mounting plate into the upper slot of the wheel mount. To attach the wheel guard, proceed as follows:

- ▶ Take the wheel guard and insert the groove-shaped cutout section of the plate into the upper groove of the of the axle block.
- ▶ Slide in the wheel guard as far as it will go. You should feel the plate engaging into the guide.
- ▶ Repeat the operation on the other side.
- ▶ For removal, pull out the wheel guard upwards.

Spoke protectors

The spoke protectors have already been installed during assembly at the factory. No subsequent modifications are possible. Should a repair of the spoke protectors become necessary, please contact your local supplier.

Floating axles for front wheels

For removing the swivel wheels, proceed as follows:

- ▶ Remove the seat unit [see section 7.1.1.).
- ▶ Fold up the SIMPLY® [see section 7.1.4.). But, do not apply the parking brake in this case.
- ▶ Lift the front of the SIMPLY® slightly, so that the front wheel is lifted from the ground.
- ▶ Press the rubber covering of the guide tube with your thumb. Thereby, a locking button located under the covering is actuated.
- ▶ Hold the button pressed and remove the swivel wheel with your other hand.
- ▶ Repeat the process on the other side.
- ▶ To attach the swivel wheels (e.g. after transportation) proceed in reverse order. The wheels must engage audibly.

Auxiliary drives

Name of the auxiliary drive	Manufacturer	Note
V-MAX	AAT	In combination with an AMF power knot system for BTW transport, it is essential to contact AAT and obtain a separate quotation for the necessary attachment parts.
CAMILINO	AAT	In combination with an AMF power knot system for BTW transport, it is essential to contact AAT and obtain a separate quotation for the necessary attachment parts.
MAX-E	AAT	In combination with an AMF power knot system for BTW transport, it is essential to contact AAT and obtain a separate quotation for the necessary attachment parts.
VIAMOBIL V25	Alber	In combination with an AMF power knot system for BTW transport, it is essential to contact Alber and obtain a separate quotation for the necessary attachment parts.
e-motion M25	Alber	In combination with an AMF power knot system for BTW transport, it is essential to contact Alber and obtain a separate quotation for the necessary attachment parts.
E-Fix E35/E36	Alber	In combination with an AMF power knot system for BTW transport, it is essential to contact Alber and obtain a separate quotation for the necessary attachment parts.

Caution!

Before using the version with floating axles, take care to make sure that the floating axles are correctly engaged.

Auxiliary drives / stair climbing aids

In the course of implementing the Medical Device Regulation (MDR), it was necessary for manufacturers of medical devices to confirm mutual compatibility insofar as products are used together. The respective intended purpose and associated intended performance of our individual products should of course not be impaired or lost in the process. This applies in particular to the attachment of auxiliary drives or stair climbing aids. The combination of a SIMPLY undercarriage with the additional drives / stair climbing aids listed below is permissible. The respective required compatibility certificate is available and can be provided at any time upon request.

Caution!

The attachment of auxiliary drives as well as stair climbing aids is only possible in connection with a stabilizer (Art. No. 73030). The creation of a connection between the upper and lower frames of the SIMPLY undercarriage is mandatory for the stabilization of the entire unit.

Caution!

For the attachment of auxiliary drives as well as stair climbing aids, only the necessary attachment parts of the respective manufacturers are to be used.

E-Drive plus 16"/22"/24"	DECON wheels	In combination with an AMF power knot system for BTW transport, it is imperative to consult Decon and obtain a separate quotation for the necessary attachments.
--------------------------	--------------	--

Caution!

If an auxiliary drive is mounted on the SIMPLY undercarriage, then the use of toggle brakes (Art. No. 73009) is mandatory. The installation of anti-tippers is mandatory.

Note!

It may be necessary to relocate the rear cross strut. This must be agreed with the manufacturer of the auxiliary drive.

Stair climbing aids

Name of the auxiliary drive	Manufacturer	Note
S-MAX	AAT	In combination with an AMF power knot system for BTW transport, it is essential to contact AAT and obtain a separate quotation for the necessary attachment parts.
SCALA-Mobil S35/S38	Alber	In combination with an AMF power knot system for BTW transport, it is essential to contact Alber and obtain a separate quotation for the necessary attachment parts.

Note!

No subsequent modifications may be made to auxiliary drives/stair climbing aids. If there is a malfunction, please contact your medical supply store. Only original spare parts may be used.

Caution!

Please ensure that you receive instructions for use for the auxiliary drive / stair climbing aid from the supplier. When using the auxiliary drive / the stair climbing aid, please have yourself trained by an employee of your medical supply store and observe the respective operating instructions at all times.

Caution!

The transport of a person sitting in a seating system, which is mounted on a SIMPLY undercarriage, including an attached auxiliary drive / stair climbing aid is not permitted.

8. Compatibility/Combination options

Basically, the interco undercarriages are designed for combination with interco seating systems. The permitted combination of an undercarriage with an interco seat shell depends on the respective sizes and can be found in the table below.

Undercarriage	SB 45 with 22"/24" wheels	SB 45 with 12"/16" wheels	SB 40 with 22"/24" wheels	SB 40 with 12"/16" wheels	SB 35 with 22"/24" wheels	SB 35 with 12"/16" wheels
ORTHOLIGHT® size 1	-	-	✓	✓	✓	✓
ORTHOLIGHT® size 2	-	-	✓	✓	✓	✓
ORTHOLIGHT® size 3	✓	✓	✓	✓	✓ without wheel cover	✓
ORTHOLIGHT® individual max. SB 32 cm	-	-	✓	✓	✓ without wheel cover	✓
ORTHOLIGHT® size 4	✓	✓	✓	✓	-	-
ORTHOLIGHT® size 5	✓	✓	✓ without wheel cover	✓	-	-
ORTHOLIGHT® size 6	✓ without wheel cover	✓ max. 80 kg	-	-	-	-

Undercarriage	SB 45 with 22"/24" wheels	SB 45 with 12"/16" wheels	SB 40 with 22"/24" wheels	SB 40 with 12"/16" wheels	SB 35 with 22"/24" wheels	SB 35 with 12"/16" wheels
ORTHOLIGHT® individual von SB 32 cm bis max. 41 cm	✓	✓	-	-	-	-
MAGICLIGHT® size 1	-	-	-	-	✓	✓
MAGICLIGHT® size 2	-	-	✓	✓	✓	✓
MAGICLIGHT® size 3	✓	✓	✓	✓	✓ without wheel cover	✓
MAGICLINE® size 1	-	-	-	-	✓	✓
MAGICLINE® size 2	-	-	✓	✓	✓ without wheel cover	✓
MAGICLINE® size 3	✓	✓	✓ without wheel cover	✓	-	-
ERGOLINE® nach Patientenmaße	✓ up to seat width 48 cm without wheel cover	✓ up to seat width 50 cm	✓ up to seat width 45 cm	✓ up to seat width 45 cm	✓ up to seat width 40 cm without wheel cover	✓ up to seat width 40 cm

In the course of implementing the Medical Device Regulation (MDR), it is necessary for manufacturers of medical devices to confirm mutual compatibility insofar as products are used together. The respective intended purpose and associated intended performance of our individual products should of course not be impaired or lost in the process. This applies in particular to the construction of seat shells from manufacturers other than interco. In these instructions for use, the manu-

facturers of seat shells are not explicitly listed. Please ask the respective manufacturer of the desired components for the compatibility certificate. Compatibility contracts for interco undercarriages can be concluded between the medical supply store and interco upon request.

The attachment of auxiliary drives and stair climbing aids is described in Chapter 7.2.4.

9. Technical data

Caution!

The maximum load of SIMPLY® with attached seat unit amounts to 80 kg.

9.1 Overall dimensions of SIMPLY®

	SB 35	SB 35	SB 40	SB 40	SB 45	SB 45
Wheels	12"/16"	22"/24"	12"/16"	22"/24"	12"/16"	22"/24"
Max. length	86 cm	91 cm	86 cm	91 cm	86 cm	91 cm
Overall width	55 cm	59 cm	60 cm	64 cm	65 cm	69 cm
Height to the upper edge of the adapter plate	50 cm					
Min. push bar height (seat angle 90°, push bar 90°)	93 cm					
Max. push bar height (seat angle 90°, push bar 90°)	103 cm					

Weight (basic version without seat unit): 20 - 22 kg, depending on the version

9.2 Tyre pressure table

	Tyre pressure (bar) maximum
Front swivel wheels	2.5
Rear wheels 12"	2.5
Rear wheels 16"	3.0
Rear wheels 22"	4.0
Rear wheels 24"	4.0

10. Transport/transport lock/climbing stairs

10.1. General information on transport

Only the tested and approved wheelchair and occupant restraint system from AMF-Bruns may be used for transporting the SIMPLY® undercarriage in motor vehicles for the transportation of the disabled. In addition, a 4-point restraint system from AMF-Bruns is required for transport in motor vehicles. For information on how to correctly secure the force application point system in the respective vehicle, please see the manufacturer's instructions and the following chapter 10.2. „Transport with an AMF force application point system“.

Warning!

Dangers due to incorrect fastening of the transport safety device

If you attach the safety device for transport in a vehicle to the wrong places on the product, it cannot be transported safely.

- ▶ Never secure the transport safety device to the frame or other unsuitable basic components!

10.2. Transport with an AMF force application point system

The SIMPLY® undercarriage was subjected to a crash test as standard version with maximum fittings in accordance with ISO 7176-19. Some models are custom-made and cannot be tested in a crash test. Nonetheless, the custom-made model is also fitted with a force application point system manufactured by AMF Bruns. Further information is available at www.amf-bruns.de.

The undercarriage is marked on four points which indicate the mounting points for the tension belts for transportation with motor vehicles. Only use the provided mounting points for strapping with tension belts. When fitted correctly, the head rest of the seat shell is very robust, but cannot replace head rests fitted to motor vehicles.

Caution!

The SIMPLY® undercarriage has been tested without an additional drive. If an additional electrical drive is to be added, then this accessory is to be disassembled if the SIMPLY® undercarriage is to be transported in a motor vehicle.

10.3. Transport on aircraft

When traveling by air, the product may only be transported without the user in the cargo hold of the aircraft.

There are no further instructions or regulations for transport in an aircraft. We therefore strongly recommend that you contact the airline before traveling by air to clarify the transport options in each individual case.

Caution!

Transport on aircraft

The user must never be in the device when it is transported in the cargo hold of an aircraft!

10.4. Climbing stairs

Stairs or high curbs can only be negotiated with the assistance of two accompanying persons. Assistants are only permitted to grip the front frame tube and the back brace of the frame, without lifting or carrying the product when doing so.

When ascending, the assistant behind the device pulls the device up the step. The assistant in the front stabilizes the position by pressing the device equally on both sides into the step.

When descending, the assistant at the front brakes by pressing the device evenly into the steps. The assistant behind holds the back brace, and stabilizes and maintains the device in the correct position. To prevent damage, slowly roll the product down the steps. Then properly readjust the anti-tipping system (if installed).

Caution!

Risks when climbing stairs

If you hold or lift the product by its attachments when climbing stairs, these may tear off and the user may be injured.

- ▶ Do not hold the product by the push bar when negotiating stairs.
- ▶ Never lift the product by the foot rest, the wheels or the arm rests or other attached parts.
- ▶ Do not lift the product with the user sitting in it.

11. Maintenance plan

11.1. Cleaning

Your interco product is made of high-quality and durable materials which are easy to clean. Please observe the following instructions for long durability of the product:

- ▶ Always keep the product dry, at temperatures between -10 and +40 °C. This prevents corrosion.
- ▶ Sand or other dirt particles should not be allowed to penetrate the product. This could damage the mechanics or - if applicable - the wheels.
- ▶ Never use the product in salt water.
- ▶ The cushions can be removed for cleaning and care.

11.1.1. Materials used

interco uses the following materials in the manufacturing of medical devices and accessories:

- ▶ Metals (aluminum, steel, stainless steel)
- ▶ Plastics (POM, PA6, PTEG)
- ▶ Beech wood (for therapy tray tables)
- ▶ Foams (frame foam, polyurethane foam, hard foam, azote foam, neoprene)
- ▶ Upholstery fabrics (mesh material, Trevira CS fabrics, artificial leather, Dartex)
- ▶ Powder coatings
- ▶ DIN and standard parts

All materials used are classified as harmless to health.

Powder-coated aluminum

interco undercarriages and some add-on parts are made of aluminum and feature a resistant plastic coating. Powder varnishes are used for coating. Powder varnishes are solvent-free coating materials, composed of different resin systems, pigments as well as special additives.

11.1.2. Cleaning of materials

Powder-coated aluminum

Clean undercarriages and attachments made of powder-coated aluminum with mild soapy water and a soft sponge. Do not use aggressive or abrasive agents. These can damage the surface of the coating.

Powder coated materials can be maintained either with lukewarm, clear water or with the addition of neutral to slightly alkaline cleaning agents. After the cleaning process, rinse with clear water.

Accessory components made of metal, plastic, wood

Please clean accessory components with a conventional mild household soap solution.

11.1.3. Disinfection of the product

Please use a water-based agent for disinfection of the product, e.g. Terralin, Quartamon Med or Sagrotan Original concentrate. Please observe the manufacturer's instructions for use when doing so. When disinfecting the fabrics, also follow the instructions of the fabric manufacturer regarding the use of brand name disinfectants.

Rinse off all detergents and disinfectants thoroughly and leave the cleaned object to dry completely before use. If this is neglected, this can lead to chemical reactions making the material brittle or lead to skin irritations of the patient.

Important!

No disinfectants for synthetic leather/skai
Synthetic leather covers/skai covers may not be cleaned with disinfecting agents, as the material gets porous over time and cracks.

Note!

Wipe disinfection

If wipe disinfection is necessary, use a disinfectant that is suitable for surface disinfection according to the DGH disinfectant list.

11.2. Maintenance

Your interco product is checked by trained and qualified employees for proper functioning and freedom from defects before being handed over to the user.

To ensure safe use of the product over its entire service life (according to the manufacturer from at least 2 years to 8 years maximum) and to ensure a perfect technical condition, it must be maintained and inspected regularly according to the following maintenance (every 6 months) and service (every 12 months) plans.

Note!

Changing settings

The changing of settings (other than those described in Section 7) is only permitted for qualified and trained personnel!

Incorrect settings by the user, operator or laypersons bear the risk of injuries or crushing, which in a worst case scenario could lead to the patient's death! During normal use, you should regularly check the interco product for damage and the perfect working condition of all components.

Important!

Original spare parts

Only original parts may be used for service and maintenance work.

Checking / components	Daily	Weekly
Visual inspection of the seating system and the attached components and accessories for visible damage	x	
Check of connection between undercarriage and seating system	x	
Visual inspection of all fastening elements	x	
Visual inspection of all accessories / options	x	
Retightening of all fastening elements		x
Retightening of all screw connections of the accessories / options		x

11.3. Service

Test / Execution	OK	Adjustments necessary
Cleanliness	<input type="checkbox"/>	<input type="checkbox"/>
Check of all settings of the SIMPLY® undercarriage	<input type="checkbox"/>	<input type="checkbox"/>
Check of setting of the foot rest:	<input type="checkbox"/>	<input type="checkbox"/>
▶ Is there enough free space at the back of the knee?		
Check the safety distances according to DIN EN 12182:2012 using the following table	<input type="checkbox"/>	<input type="checkbox"/>

Safety distances according to DIN EN 12182:2012 for children

To avoid	Between moving parts	Between fixed parts
Trapping fingers	<4 mm or >25 mm	<5 mm or >12 mm
Trapping feet	<25 mm or >120 mm	<25 mm or >45 mm
Trapping the head	<60 mm or >300 mm	<60 mm or >250 mm
Trapping the genitals	<8 mm or >75 mm	<8 mm or >75 mm

Safety distances according to DIN EN 12182:2012 for adults

To avoid	Between moving parts	Between fixed parts
Trapping fingers	<8 mm or >25 mm	<8 mm or >25 mm
Trapping feet	<35 mm or >120 mm	<35 mm or >100 mm
Trapping the head	<120 mm or >300 mm	<120 mm or >250 mm
Trapping the genitals	<8 mm or >75 mm	<8 mm or >75 mm

Note!

Expiry of the manufacturer's warranty if service is not conducted

The product must be checked regularly – at least once a year – as part of a (payable) service.

Otherwise interco Group cannot provide any warranty.

11.3.1. Seating system

Check of setting / function	OK	Damage/distortion	Exchange/replacement
Main frame	Main frame	<input type="checkbox"/>	<input type="checkbox"/>
	Anti-tipping system	<input type="checkbox"/>	<input type="checkbox"/>
Trapezoidal adapter	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Front wheel	Wheels	<input type="checkbox"/>	<input type="checkbox"/>
	Tire pressure	<input type="checkbox"/>	<input type="checkbox"/>
	Running performance of the wheels	<input type="checkbox"/>	<input type="checkbox"/>
	Front wheel fork	<input type="checkbox"/>	<input type="checkbox"/>
	Mounting to frame	<input type="checkbox"/>	<input type="checkbox"/>
Rear wheel	Wheels	<input type="checkbox"/>	<input type="checkbox"/>
	Tire pressure	<input type="checkbox"/>	<input type="checkbox"/>
	Running performance of the wheels	<input type="checkbox"/>	<input type="checkbox"/>
	Quick-release axles	<input type="checkbox"/>	<input type="checkbox"/>
	Drum brake	<input type="checkbox"/>	<input type="checkbox"/>
Brakes	Performance test, level 1	<input type="checkbox"/>	<input type="checkbox"/>
	Performance test, level 2	<input type="checkbox"/>	<input type="checkbox"/>
	Performance test, level 3	<input type="checkbox"/>	<input type="checkbox"/>
	Bowden cable	<input type="checkbox"/>	<input type="checkbox"/>
Adjustability	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Gas-pressure springs	Damages	<input type="checkbox"/>	<input type="checkbox"/>
	Screw connections	<input type="checkbox"/>	<input type="checkbox"/>

Check of setting / function		OK	Damage/ distortion	Exchange/ replacement
Seat adjustment	Performance test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Folding mechanism	Folding mechanism	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Locking mechanism	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Back angle adjustment	Performance test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Push bar	Height adjustment mechanism	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Locking mechanism	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Screw connections	Size and length	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Completeness	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Damages	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Loctite secured	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Covering caps available	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Options	Wheel protector	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Spoke protector	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	E-Fix	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Centered foot rest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Appearance (visual inspection)		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Paintwork		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Cleanliness		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Rating plate		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

12. Reuse/durability

Your product can be reused under certain conditions. It is CE marked and must be reconditioned for reuse by the manufacturer to meet all essential safety and performance requirements. This includes that it is properly adapted to the needs of the new user. In particular, individual fittings can only be reinstated after professional adaptation to the new patient. Accompanying documents, as well as these operating instructions, are part of the product and must be handed over to the new user when it is to be reused again. The durability and service life of the product are not limited and

may vary. They depend on the conditions of use and the environment. In addition, daily use and the degree of disability can affect the durability and service life of the product. Regular inspection and the performance of all necessary service and maintenance work can positively influence the durability and service life of the product.

The product service life depends on the usage and environmental conditions and can be assumed to be 2 - 8 years.

13. Disposal/environmental compatibility

Environmental protection is important. Therefore interco does not work with hazardous substances and tries to integrate environmental aspects into its daily work and thinking and to pass these approaches on to our suppliers if necessary. All employees are frugal in using the resources provided to us.

All packaging materials are recyclable. interco participates in the Dual System so that environment-friendly disposal of transport packaging is ensured. The product can be broken down into its individual material components at the end of its useful life. Dispose of these raw materials, depending on their type, in accordance with the applicable environmental protection regulations.

14. Possible faults

All serious incidents and faults occurring in connection with the product and its use must be reported immediately to the specialist dealer. Furthermore, you can inform the Compliance Officer of interco and the competent authority in your country.

Your Compliance Officer at interco:
Ute Markwald (E-mail: msib@interco-reha.de)

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices

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Artikel 19 MDR
Artikel 19 MDR

Eitorf, 2022-08-12
Gültig bis 2024-08-11
Valid till

Michael Markwald
CEO Chief Executive Officer

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5
April 2017 on medical devices

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Artikel 19 MDR
Artikel 19 MDR

Eitorf, 2022-08-12

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Michael Markwald
CEO *Chief Executive Officer*

16. Warranty policy

In addition to our General Terms and Conditions, we grant a two-year warranty on all products supplied by us. Unusable or defective parts will be repaired or replaced free of charge during the warranty period. The warranty shall exclude design-related modifications.

interco Group does not assume any liability for damages resulting from the combination of our products with third-party products of any kind, unless our company has given its express approval for such a product combination.

Furthermore, warranty also excludes any defects occurring as the result of normal wear, excessive stress, violent and willful damage, as well as inappropriate use. Warranty obligations shall cease to apply if original interco replacement parts are not used for repairs and if repairs and modifications are not performed by interco. The installation of additional parts or changes to the interco products by others than the manufacturer are not permitted and can be subject to legal

action if they become known. Likewise, the warranty obligations of interco Group shall cease to apply in such cases and any liability claims directed at the manufacturer shall no longer be valid.

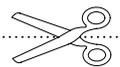
Certain screws and nuts on your interco product may become loose when the product is used. Re-tighten these at periodic intervals or arrange to have this done by your local dealer or by interco.

Note!

Expiry of the manufacturer's warranty
The product must be checked regularly – at least once a year – as part of a (payable) service. Otherwise interco Group cannot provide any warranty.

interco cannot grant warranty if third party components (not manufactured by interco) are used without authorization by interco.

Warranty extension

On receipt of this form, we will grant an extended warranty for your SIMPLY® for a total of two years. Please fill out this warranty card carefully and completely and return it to us. Thank you very much.

Funding agency/health insurance fund: _____

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Status as per September 2022 | Subject to technical alterations without prior notification

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Management system certified as per

DIN EN ISO 13485:2016

Reg.-Nr. 73 105 6351

TÜV Hessen

DIN EN ISO 9001:2015

Reg.-Nr. 73 100 6351

TÜV Hessen

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© interco Group GmbH, Eitorf, 2022
interco Group GmbH